


Micro-Needle

Blood collection from your smallest patients




Micro-Needle


- Minimal blood loss with no residual volumes in the adapter
- Ridged grip for excellent needle control
- Rotatable by 360° for fragile veins
- Optimal flow thanks to exceptional needle quality and exposed outlet

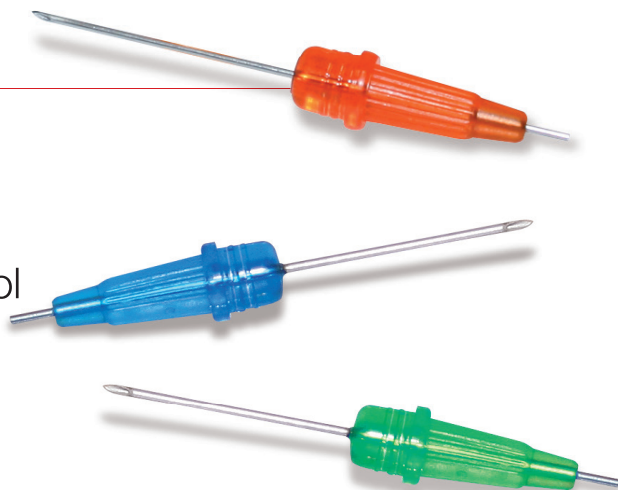

Patient-friendly blood collection from neonates and premature babies

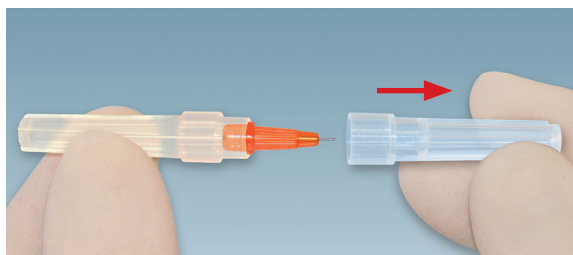

1 Remove cap.

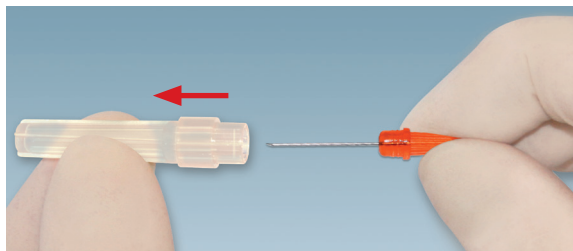

2 Withdraw Micro-Needle from protective sleeve.



3 Perform venipuncture and let blood drip into a suitable micro tube.



4 Dispose of Micro-Needle in a suitable sharps container.

Ordering Information

Order No.	Description	Ø / Length	Packaging
85.921	Micro-Needle 21G x 3/4", green	0.8 mm / 36 mm	50 / inner box, 500 / case
85.923	Micro-Needle 23 G x 3/4", blue	0.6 mm / 36 mm	50 / inner box, 500 / case
85.925	Micro-Needle 25 G x 3/4", orange	0.5 mm / 36 mm	50 / inner box, 500 / case

Sarstedt Inc.
1025 St. James Church Road
P.O. Box 468
Newton, NC 28658 - 0468
Tel: +1 800 257 5101
Tel: +1 828 465 4000
Fax: +1 828 465 4003
customerservice.us@sarstedt.com
www.sarstedt.com