


S-Monovette® EDTA

Wichtige Information zur Handhabung

Zur Vermeidung von Fehlmessungen bzw. Nichtbearbeitung der Proben aufgrund Unterfüllung sollte das **Füllvolumen mindestens 80%*** betragen, da sonst die Werte verfälscht werden.

Diese Fehler lassen sich vermeiden, wenn bei der Blutentnahme die Kolbenstange bis zum Anschlag zurückgezogen wird und danach noch ca. 5 sec. gewartet wird, bis der Blutfluss sichtbar stoppt.

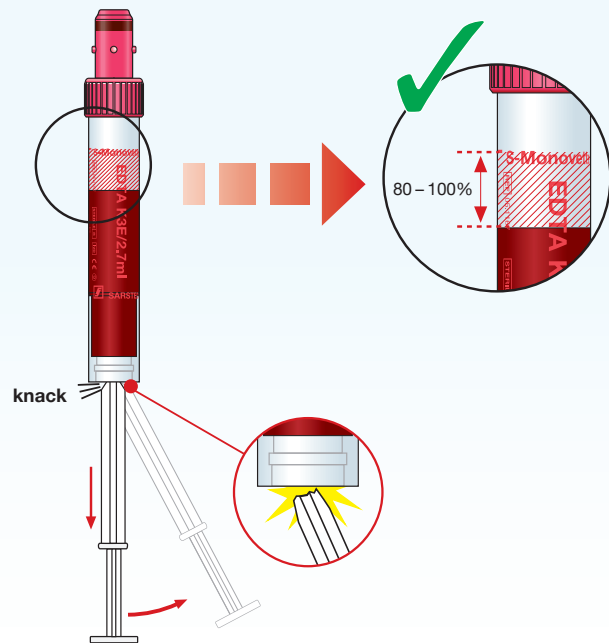


*gemäß ISO 6710 (1,2-2,0mg EDTA/ml Blut)

Barcode-Etikettierung


richtig

falsch

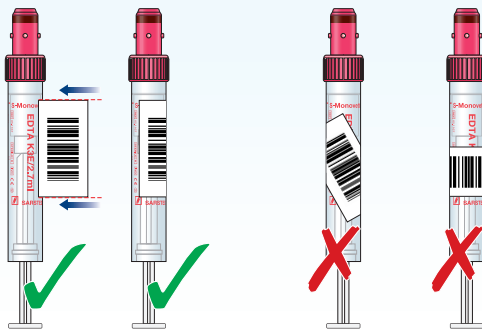

Wichtig: Gut mischen!

Die Blutprobe direkt nach der Entnahme **mindestens 3x** über Kopf schwenken!

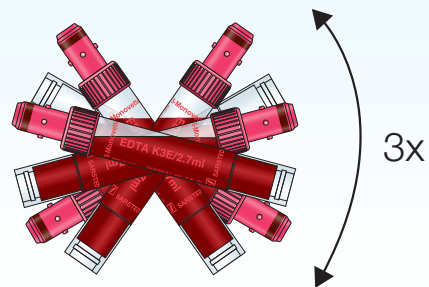

Probenstabilität

Stabilität ist instrumentenabhängig.
Ausstrich **innerhalb von 3 Stunden** nach Probennahme anfertigen.

W.G. Guder et. al. ; Die Qualität diagnostischer Proben; 7. Auflage 2012

EDTA-Blut nicht im Kühlschrank lagern!!



SARSTEDT AG & Co. KG
Postfach 12 20 · D-51582 Nümbrecht
Telefon: +49 2293 305 0
Telefax: +49 2293 305 3450
Kundenservice Deutschland
Telefon 0800 0 83 305 0
info@sarstedt.com
www.sarstedt.com


 **SARSTEDT**

S-Monovette® EDTA

Important information for use

In order to avoid incorrect measurements or samples not processed due to underfilling, **the filling volume should be at least 80%***. Otherwise the values will be falsified.


To avoid these errors during blood collection, pull back the plunger until it locks into the tube base and then wait approx. 5 seconds until the blood flow visibly stops.

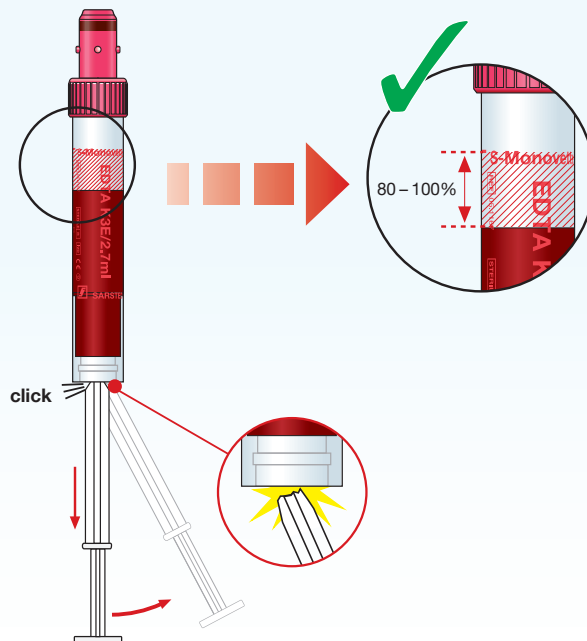


*as per ISO 6710 (1.2-2.0 mg EDTA/ml blood)

Barcode labelling

correct

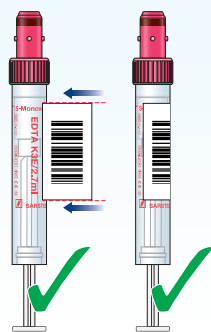


wrong

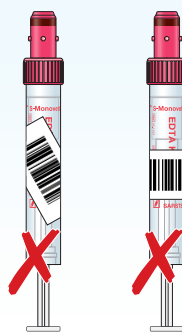

Important: Mix thoroughly!


After blood collection, gently invert the blood sample **at least three times**.

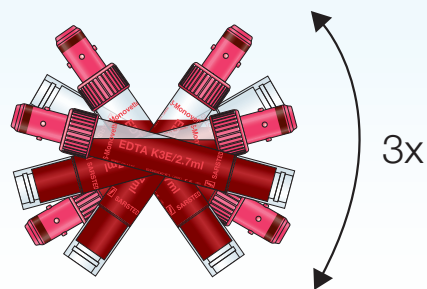

Sample stability

Stability is instrument-dependent. Prepare the smear **within 3 hours** after blood collection.

Do not store EDTA blood in a refrigerator!!

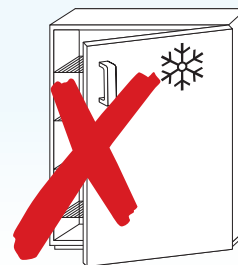

W.G. Guder et. al. ; Die Qualität diagnostischer Proben; 7. Auflage 2012

SARSTEDT AG & Co. KG
P.O. Box 12 20
D-51582 Nümbrecht
Phone: +49 2293 305 0
Fax: +49 2293 305 3992
export@sarstedt.com
www.sarstedt.com

 **SARSTEDT**