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SARSTEDT Urin-Monovette® Z

DE	Gebrauchshinweis – SARSTEDT Urin-Monovette® Z	2 - 6
EN	Instructions for Use – SARSTEDT Urine Monovette® Z	7 - 11
BG	Инструкции за употреба – SARSTEDT Monovette® Z за урина	12 - 16
CS	Návod k obsluze – Zkumavka Urine Monovette® Z SARSTEDT	17 - 21
DA	Brugsanvisning – SARSTEDT urin-Monovette® Z	22 - 26
EL	Οδηγίες χρήσεως – Monovette® Z ούρων SARSTEDT	27 - 31
ES	Instrucciones de uso – Monovette® Z de orina SARSTEDT	32 - 36
ET	Kasutusjuhend – Uriinikogumissüsteem SARSTEDT Urin-Monovette® Z	37 - 41
FR	Mode d'emploi – Monovette® Urine Z SARSTEDT	42 - 46
HR	Uputa za upotrebu – SARSTEDT epruveta za urin Urin-Monovette® Z	47 - 51
HU	Használati utasítás – SARSTEDT vizelet Monovette® Z	52 - 56
IT	Istruzioni d'uso - SARSTEDT Monovette® Z urina	57 - 61
KO	사용 설명서 – SARSTEDT 소변 Monovette® Z	62 - 66
LT	Naudojimo instrukcijos – SARSTEDT „Urine Monovette® Z“	67 - 71
LV	Lietošanas norādes – SARSTEDT urīna Monovette® Z	72 - 76
NL	Gebruiksaanwijzing – Urine-Monovette® Z van SARSTEDT	77 - 81
NO	Bruksanvisning – SARSTEDT Urin-Monovette® Z	82 - 86
PL	Instrukcja obsługi – Probówka do pobierania moczu Urin-Monovette® Z SARSTEDT	87 - 91
PT	Instruções de utilização – Monovette® Z para urina SARSTEDT	92 - 96
RO	Instrucțiuni de utilizare - Monovette® urinară SARSTEDT Z	97 - 101
RU	Инструкция по применению – SARSTEDT Urin-Monovette® Z	102 - 106
SK	Návod na Použitie – SARSTEDT skúmavka Urin-Monovette® Z	107 - 111
SL	Navodila za uporabo – Urinska epruveta Urin-Monovette® Z SARSTEDT	112 - 116
SV	Bruksanvisning – SARSTEDT urin-Monovette® Z	117 - 121
TR	Kullanım için talimatlar – SARSTEDT Urin-Monovette® Z	122 - 126

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DIN EN 14254: In vitro diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans

CLSI* GP16 "Urinalysis Approved Guideline"

*CLSI (Clinical and Laboratory Standards Institute)

European Urinalysis Guidelines, Scand J Clin Lab Invest 2000; 60: 1-96

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Intended use

The Urine Monovette® Z is used as a specimen container for the collection, transport, processing (e.g. by centrifugation) and storage of urine samples for *in vitro* diagnostic tests. It can be stored at room temperature or cooled at up to -20 °C depending on the stability of the parameters to be tested. The product is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The Urine Monovette® Z, except for the version with light protection properties, consists of a transparent plastic container, a conical plunger with plunger rod, a coloured Luer screw cap with stopper and an enclosed suction tip. Each Urine Monovette® Z has either a plastic or a paper label. In addition, various versions of the Urine Monovette® Z are available for different sample volumes. They do not contain any stabilising additives. The white Urine Monovette® Z with light protection properties, Art. no. 10.252.030, is especially suitable for protecting light-sensitive parameters. Sterile and non-sterile versions are available.

Name	Dimensions
Urine Monovette® Z • 3.2 ml	3.2 ml 75 x 13 mm
Urine Monovette® Z • 8.5 ml	8.5 ml 15 x 92 mm
Urine Monovette® Z • 10 ml	10 ml 102 x 15 mm

Safety information and warnings

NOTE FOR STERILE VERSIONS: DO NOT USE THE PRODUCT IF THE BLISTER PACKAGING IS DAMAGED.

- General precautions:
Use gloves and other general personal protective equipment to protect yourself against urine and possible exposure to pathogens transmitted by biological samples.
- Handle all biological samples and sharp collecting utensils (needles) in accordance with the guidelines and procedures in your facility. In case of direct contact with biological specimens or a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases. Always follow the safety guidelines and procedures of your facility.
- The product is intended for single use. Dispose of the product and all collection utensils in containers for hazardous biological waste.
- Do not use the product after the expiry date. The expiry date of the Urine Monovette® Z is the last day of the month and year indicated.

Storage

Store the product at room temperature.

Transport

The product is a primary receptacle according to the ADR (Packing instruction P650) and IATA regulations.

Limitations

The storage period and temperature for a filled Urine Monovette® Z depend on the expiry date of the parameters for investigation.

Urine analysis should be carried out within one to two hours. To potentially prolong sample stability for some analytes, the urine sample can be cooled at 2-8 °C. There are exceptions, for example for bilirubin and urobilinogen. To protect these and other light-sensitive parameters, the Urine Monovette® Z with light protection properties can be used. Cooling can promote the precipitation of amorphous urates or phosphates in the urine sample and impair urine microscopy.

Do not freeze urine samples if they are to be used for urine sediment.

Collecting and handling specimens

READ THIS DOCUMENT COMPLETELY BEFORE STARTING A URINE TRANSFER.

- Gloves, gown, goggles or other suitable protective wear to protect against pathogens or potentially infectious materials transmitted by urine.
- Required number of Urine Monovettes.
- Only for collection from an indwelling bladder catheter:
Required material for disinfecting the patient collection point on the urinary drainage system (observe facility guidelines on setting up and preparing the urine sample collection site).
- Stand or other suitable device for placing the filled Urine Monovettes.
- Disposal container for hazardous biological waste.

Recommended order of collection

If multiple Urine Monovettes are to be filled from one primary container, the following order of collection is recommended:






Urine Monovette®	Intended use
1. Urine Monovette® Z (without stabiliser)	e.g. strip tests, sediment or albumin-creatinine ratio
2. Urine Monovette® Z (without stabiliser)	Microbiological tests
3. Urine Monovette® Boric acid (with stabiliser)	
4. Urine Monovette® Z (without stabiliser)	Quantitative chemical tests (Urine Monovette® Z not used in step 1)

Procedure for transferring the urine specimen to a Urine Monovette® Z

A: REMOVAL OF THE URINE SAMPLE FROM A URINE CUP

Prepare the urine sample for withdrawal by carefully swivelling the urine cup and then open the cup.






- Only for individually sterile-wrapped Urine Monovettes:
Open the blister package with the thumb and index finger using the peel-off technique. The transparent film must be facing upwards.
- Hold the Urine Monovette® Z upright with the tip pointing upwards and remove the stopper on the Luer screw cap **a**. Set it aside for later. Place a suction tip on the Luer connection and make sure the suction tip is firmly attached **b**.
- Now dip the suction tip into the urine cup and withdraw the plunger rod until the plunger has reached the base line and the Urine Monovette® Z is filled with urine **c**. Take the Urine Monovette® Z out of the liquid and hold it upright with the suction tip pointing upwards **d**. To empty the suction tip, pull the plunger rod downwards to the stop **e**.
- Remove the suction tip and dispose of it in a disposal container for hazardous biological waste **f**.
- Close the Urine Monovette® Z securely with the stopper set aside previously **g**. Only for Urine Monovette® Z 3.2 ml: Pull the plunger rod down until it audibly clicks into the final position **h**. Break off the plunger rod **i**. Place the Urine Monovette® Z in a stand.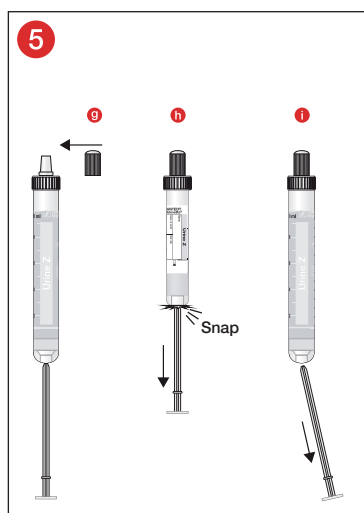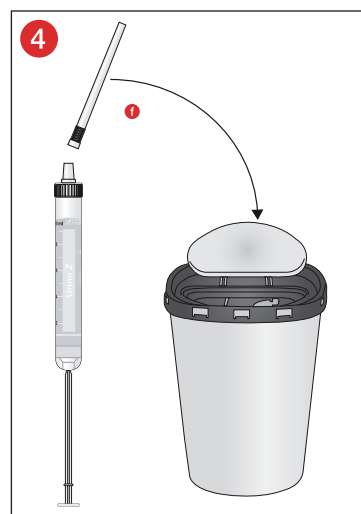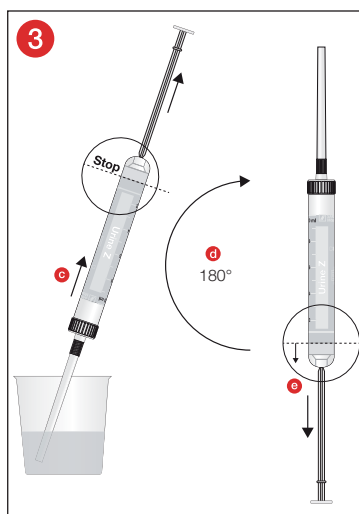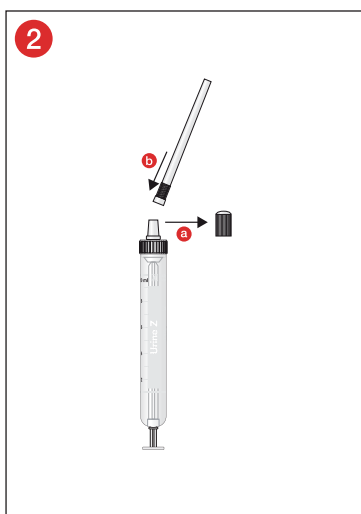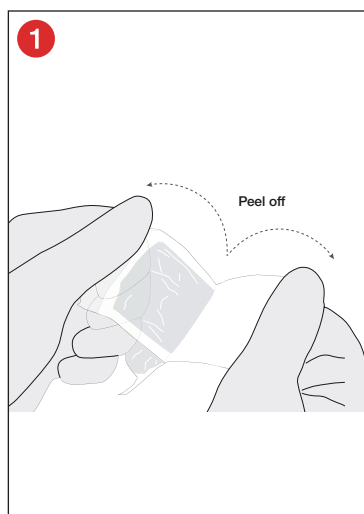

B: REMOVAL OF THE URINE SAMPLE FROM AN INDWELLING BLADDER CATHETER

Carry out the preparations for collecting the urine sample at the urine sample collection site following the guidelines at your facility.

- 1 Only for individually sterile-wrapped Urine Monovettes:
Open the blister package with the thumb and index finger using the peel-off technique. The transparent film must be facing upwards.
- 2 Hold the Urine Monovette® Z upright with the tip pointing upwards and remove the stopper on the Luer screw cap **a**. Set it aside for later.
- 3 Penetrate the septum of the urine sample collection site and ensure the Urine Monovette® Z is firmly secured with one hand **b**.
- 4 Now withdraw the plunger rod with the other hand back to the stop until the Urine Monovette® Z is filled with urine **c**.
Remove the Urine Monovette® Z from the urine sample collection site and hold it upright with the opening pointing upwards **d**.
- 5 Close the Urine Monovette® Z securely with the stopper set aside previously **e**.
Only for Urine Monovette® Z 3.2 ml: Pull the plunger rod down until it audibly clicks into the final position **f**.
Break off the plunger rod **g**.
Place the Urine Monovette® Z in a stand.

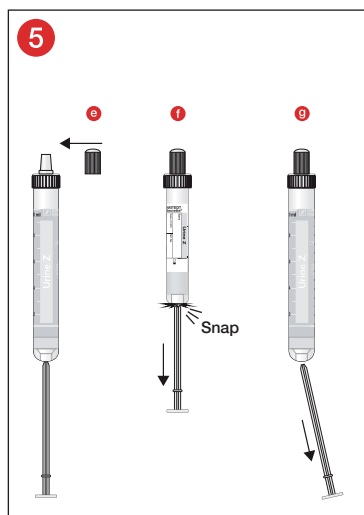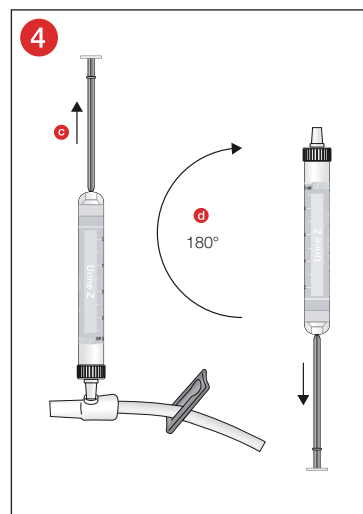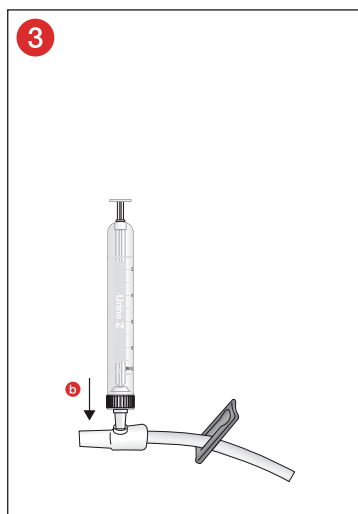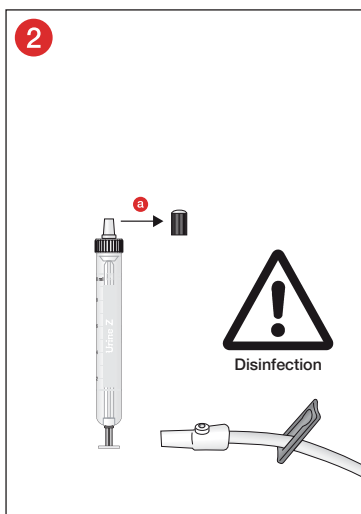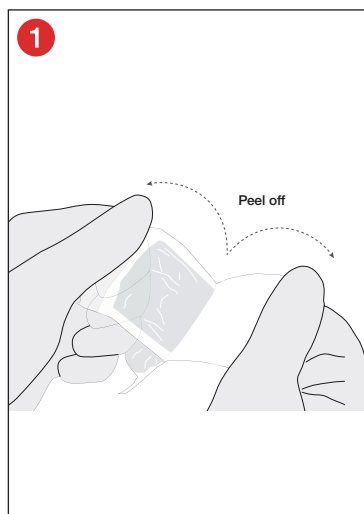Freezing / Thawing

THE FOLLOWING GENERAL RECOMMENDATIONS FOR FREEZING APPLY TO URINE MONOVETTES:

Freezing below 0 °C

- Check beforehand whether freezing will adversely affect the urine sample or analysis (parameter stability, interference: e.g. haemolysis). Important! Do not freeze urine samples if they are to be used for urine sediment.

NOTE: For the stability of the parameters, consult the instructions for use of the test reagents / analysis device manufacturers.

- The strength values of plastics are reduced in the temperature range below 0 °C. Mechanical stress should therefore be avoided.
- The conditions for freezing must be selected so that the contents of the Urine Monovette® freeze evenly or from bottom to top. The Urine Monovette® should have enough room in a holder or storage box to allow it to expand. Holders made of Styrofoam or metal are unsuitable as they can lead to expansion tears.

Freezing at -20 °C

- Cool the Urine Monovette® in an upright position for a period of 45-60 min. from room temperature to +4 °C before it can be frozen at -20 °C.

Freezing below -20 °C

- Deep-freezing to below -20 °C has not been tested by the manufacturer. Due to the number of possible influencing factors, it is recommended to conduct freezing tests under routine laboratory conditions.

Thawing

- Allow the Urine Monovette® to thaw for at least 45 min. at room temperature in an upright position. Mechanical stress must also be prevented in this step. Thawing too quickly can impair the results of the analysis.

Centrifugation

IMPORTANT! SARSTEDT Urine Monovettes are designed for maximum 3000 x g (RCF - relative centrifugal force – g-force).

Centrifuge inserts should be selected according to the size of the Urine Monovettes used. The relative centrifugal force has the following ratio to the selected rpm:

$$\text{RCF (g-force)} = 11.2 \times r \times (\text{rpm}/1000)^2$$

"RCF": "Relative centrifugal force"

"rpm": "Revolutions per minute",

"r" [in cm]: "Radius of rotation from the middle of the centrifuge to the bottom of the Urine Monovette®."

Use suitable tube holders or inserts only. Centrifuging Urine Monovettes with cracks or centrifuging at an excessive centrifugal force can lead to breakage of the Urine Monovettes, which can release potentially hazardous substances.

Urine Monovettes should be centrifuged in accordance with the centrifugation conditions listed below. If other conditions are used, they must be validated by the user themselves.

It must be ensured that the Urine Monovettes fit well in the centrifuge inserts. Urine Monovettes that protrude above the insert can get caught in the centrifuge head and be broken. Ensure that the centrifuge is filled evenly. Please observe the instructions for use of the centrifuge.

CAUTION! Do not remove broken Urine Monovettes by hand.

Information on disinfecting the centrifuge can be found in the instructions for use of the centrifuge.

Centrifugation recommendation:

The recommended RCF force (g-force) for urine samples for sediment extraction is 400 x g for 5 minutes.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled Urine Monovettes must be disposed of in suitable containers for hazardous biological waste.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Product-specific standards and guidelines as amended/updated

DIN EN 14254: In vitro diagnostic medical devices – Single-use receptacles for the collection of specimens, other than blood, from humans

CLSI* GP16 "Urinalysis Approved Guideline"

*CLSI (Clinical and Laboratory Standards Institute)

European Urinalysis Guidelines, Scand J Clin Lab Invest 2000; 60: 1-96

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Store away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

The following also applies to sterile products:

	Sterilisation by irradiation
	Single sterile barrier system with external protective packaging
	Do not use if package is damaged
	Do not resterilise

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