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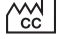
DIN EN 14254: *In vitro* diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans

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Intended use

The V-Monovette® VD specimen collection container is used to collect saliva samples in a gargle solution and aids in the collection, transport, processing and storage of saliva samples in a gargle solution for *in vitro* diagnostics (e.g. SARS CoV-2 detection and other virus diagnostics).

The VD specimen collection cup with transfer unit is used to collect saliva samples in a gargle solution and aids in collection with the V-Monovette® VD.

The products are available individually and can be used together. The products are intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The V-Monovette® VD and VD cup with transfer unit consist of plastic containers. The V-Monovette® VD has a light blue stopper and the product has a defined vacuum corresponding to the nominal volume. The VD cup with transfer unit has a light blue screw cap with integrated transfer unit. The products do not contain any preparations.

Name	Dimensions
V-Monovette® VD 10 ml	10 ml 100 x 15 mm
V-Monovette® VD 4 ml	4 ml 75 x 13 mm
VD cup with transfer unit	max. 100 ml 73 x 62 mm

Safety information and warnings

1. General precautions: Use gloves and general personal protective equipment to protect yourself from possible exposure to potentially infectious specimen material and transmitted pathogens.
2. Handle all biological specimens and collection utensils according to the guidelines and procedures in your facility. In the event of contact with biological samples, consult a doctor, as this can transmit viral diseases or other infectious diseases. Always follow the safety guidelines and procedures of your facility.
3. Dispose of the VD cup with transfer unit in a disposable container and all other collection utensils in containers for hazardous biological waste.
4. Do not use the products after the expiry date. The expiry date of the V-Monovette® VD and VD cup with transfer unit is the last day of the month and year indicated.
5. To prevent needlestick injuries, never put fingers in the transfer unit of a VD cup with transfer unit.

Storage

Store the products at room temperature.

Transport

V-Monovette® VD: The products can be transported together with an ADR-compliant (Packing instruction P650) secondary vessel with suction insert.

Limitations

The storage period and temperature for a filled V-Monovette® VD depend on the expiry date of the parameters for investigation. Therefore, and because of the onset of bacterial growth in saliva after a few hours at room temperature, it is recommended to analyse the firmly closed V-Monovette® VD immediately after collecting the specimen or to store it in the refrigerator until further processing.

These products may not be used for children under the age of 3 or for persons with a higher risk of choking.

Collecting and handling specimens










READ THIS DOCUMENT COMPLETELY BEFORE STARTING A GARGLE SOLUTION TRANSFER.

Material needed for the gargle solution transfer:

1. Gloves, gown, goggles or other suitable protective wear to protect against potentially infectious materials.
2. Required number of SARSTEDT V-Monovettes VD and VD cups with transfer unit.
3. Disposal container for hazardous biological material for the safe disposal of used materials and disposal container for disposing of used VD cups with transfer units.

Intended use:

- 1 Note: Before collecting saliva, the patient should not eat, drink, brush their teeth, take medication or smoke for at least 60 min before the test.
- 2 Unscrew the VD cup with transfer unit and give the cup to the patient without the lid and transfer unit.
- 3 The patient gargles with a gargle solution selected by the doctor.
- 4 The patient spits the gargle solution into the cup. At least 9 ml liquid are needed (*observe laboratory instructions*).
- 5 After transferring the gargle solution, close the cup again with the lid and transfer unit.
- 6 Remove the safety label from the top of the cup to allow access to the integrated transfer unit. Stick the safety label back on the opening to reseal it after transferring the gargle solution.
- 7 NOTE: Do not touch the transfer unit.
- 8 Guide the V-Monovette® VD with the cap pointing downwards into the integrated transfer unit and make sure that the needle penetrates the stopper of the V-Monovette® VD.
The V-Monovette® VD starts filling automatically according to the defined vacuum. If the gargle solution does not flow out or if the flow of gargle solution stops before reaching a sufficient fill level, the following steps are recommended:
a. Press the V-Monovette® VD forward again firmly to ensure that the stopper has been completely penetrated.
b. Secure the V-Monovette® VD in the transfer unit during the filling process by pressing on the bottom of the V-Monovette® VD.
c. If the gargle solution still does not flow, remove the V-Monovette® VD and insert a new V-Monovette® VD.
Hold the V-Monovette® VD in place until no more gargle solution flows into it and only then remove it from the transfer unit.
NOTE: The minimum filling quantity required for filling a V-Monovette® VD is 9 ml for the closed cup.
- 9 Dispose of the VD cup with transfer unit in a container for hazardous biological waste.
- 10 The V-Monovette® VD is ready for analysis.

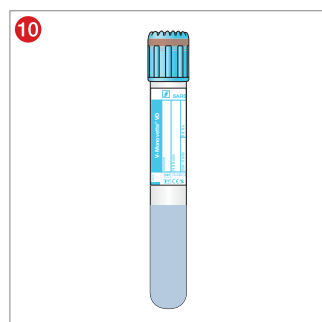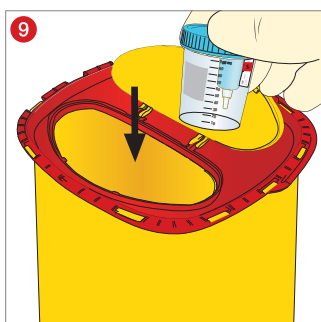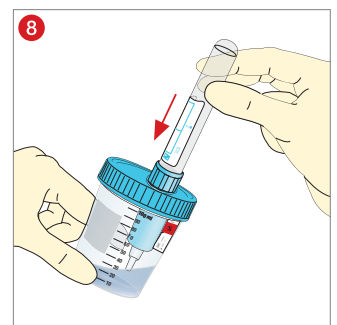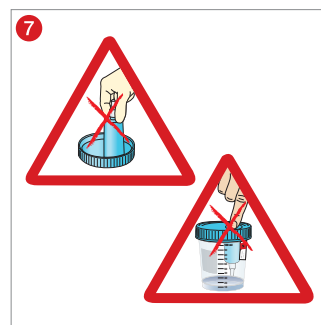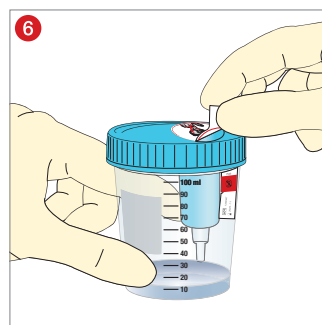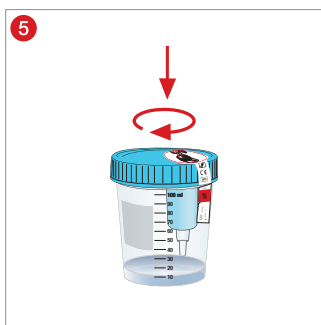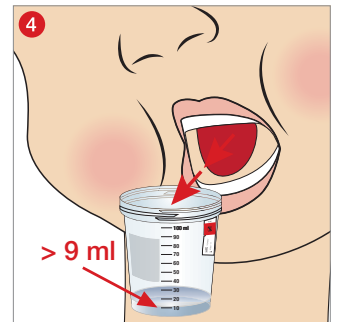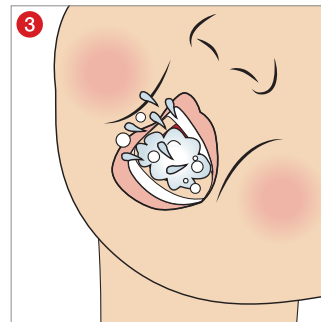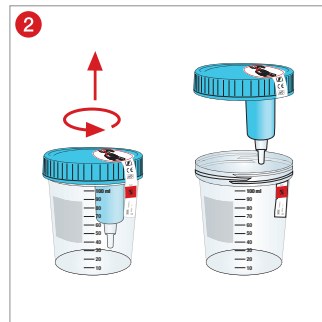Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves reduce the risk of infection.
3. Contaminated or filled V-Monovettes and used VD cups with transfer unit must be disposed of in suitable containers for hazardous biological waste.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Product-specific standards (as amended/updated)

DIN EN 14254: *In vitro* diagnostic medical devices – Single-use receptacles for the collection of specimens, other than blood, from humans

Key for symbols and labels:

	Article number
	Batch number
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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