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SARSTEDT Liquor/CSF-Monovette®

DE	Gebrauchshinweis – SARSTEDT Liquor/CSF-Monovette®	2 - 5
EN	Instructions for use – SARSTEDT CSF Monovette®	6 - 9
BG	Инструкции за употреба – SARSTEDT Monovette® за ликвор / ЦСТ	10 - 13
CS	Návod k obsluze – Zkumavka na likvor/CSF-Monovette® SARSTEDT	14 - 17
DA	Brugsanvisning – SARSTEDT Liquor/CSF-Monovette®	18 - 21
EL	Οδηγίες χρήσεως – SARSTEDT Monovette® για εγκεφαλονωτιαίο υγρό/ENY	22 - 25
ES	Instrucciones de uso – CSF-Monovette® SARSTEDT de líquido cefalorraquídeo	26 - 29
ET	Kasutusjuhend – SARSTEDT Liquor/CSF-Monovette®	30 - 33
FR	Mode d'emploi – Monovette® pour LCR/LCS	34 - 37
HR	Uputa za upotrebu – epruveta SARSTEDT Liquor/CSF-Monovette®	38 - 41
HU	Használati utasítás – SARSTEDT Liquor/CSF-Monovette®	42 - 45
IT	Istruzioni d'uso – Monovette® Liquor/LCR SARSTEDT	46 - 49
KO	사용 설명서 – SARSTEDT 뇌척수액 / CSF Monovette®	50 - 53
LT	Naudojimo instrukcijos – SARSTEDT Liquor/CSF-Monovette®	54 - 57
LV	Lietošanas norādes – SARSTEDT Likvora/CSF-Monovette®	58 - 61
NL	Gebruiksaanwijzing – SARSTEDT Liquor/CSF-Monovette®	62 - 65
NO	Bruksanvisning – SARSTEDT Liquor/CSF-Monovette®	66 - 69
PL	Instrukcja obsługi – próbówki Monovette® na płyn mózgowo-rdzeniowy (PMR) SARSTEDT	70 - 73
PT	Instruções de utilização – Monovette® para liquor/LCR SARSTEDT	74 - 77
RO	Instrucțiuni de utilizare – Monovette® pentru lichid/LCR SARSTEDT	78 - 81
RU	Инструкция по применению – SARSTEDT Monovette® для спинномозговой жидкости (СМЖ)	82 - 85
SK	Návod na Použitie – Skúmavka SARSTEDT Liquor/CSF-Monovette®	86 - 89
SL	Navodila za uporabo – Epruveta za likvor/CSF-Monovette® SARSTEDT	90 - 93
SV	Bruksanvisning – SARSTEDT likvor/CSF-Monovette®	94 - 97
TR	Kullanım için talimatlar – SARSTEDT Liquor/BOS Monovette®	98 - 101

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Intended use

The CSF Monovette® is used as a sample tube for the collection, transport, processing (e.g. by centrifugation) and storage of CSF (cerebrospinal fluid) samples from a primary receptacle for *in vitro* diagnostic tests. It can be stored at room temperature or cooled down to -20 °C depending on the stability of the parameters to be tested. The product is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The CSF Monovette® consists of a transparent plastic container, a conical plunger with plunger rod and a coloured Luer screw cap with stopper. The CSF® Monovette is provided with a paper label and is sterile. No stabilising additives are contained.

Name	Dimensions
CSF Monovette® 10 ml	10 ml 102x15 mm

Safety information and warnings

1. General precautions: Use gloves and other general personal protective equipment to protect yourself against CSF and possible exposure to pathogens transmitted by biological samples.
2. Handle all biological specimens and sharp collection utensils (needles) according to the guidelines and procedures of your facility. In case of direct contact with biological specimens or a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases. Always follow the safety guidelines and procedures of your facility.
3. Dispose of all sharp objects for CSF collection in appropriate sharps containers.
4. Check each individual package for damage before use and do not use if packaging is damaged.
5. The product is intended for single use. Dispose of the product in suitable disposal containers for hazardous biological waste.
6. Do not use the product after the expiry date. The expiry date of a CSF Monovette® is the last day of the month and year indicated.

Storage

Store the product at room temperature.

Transport

The product is a primary receptacle according to the ADR (Packing instruction P650) and IATA regulations.

Limitations

The storage period and temperature for a filled CSF Monovette® will depend on the expiry date of the parameters for investigation. Check the CSF sample quality. Note the information provided by the test manufacturer regarding suitability for haemolysis and precipitates for clinical tests.

Collecting and handling specimens

READ THIS DOCUMENT COMPLETELY BEFORE STARTING A CSF SAMPLE TRANSFER.

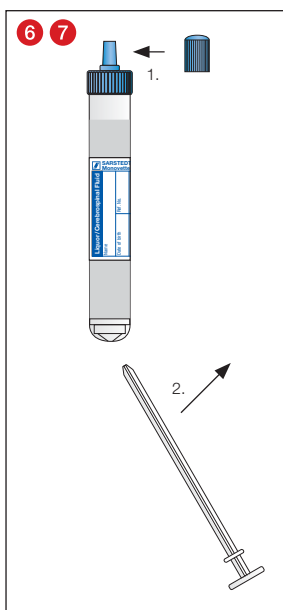
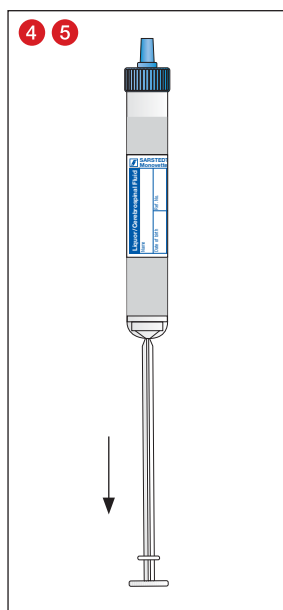
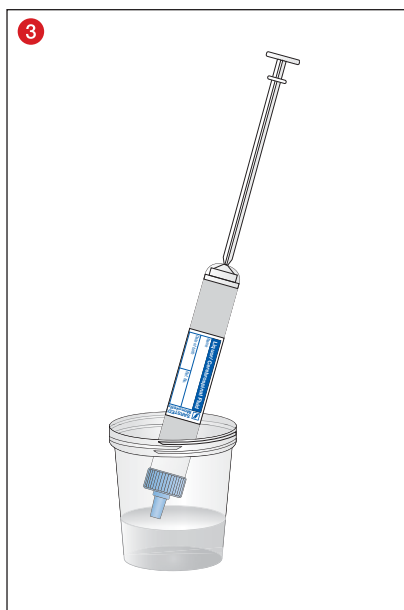
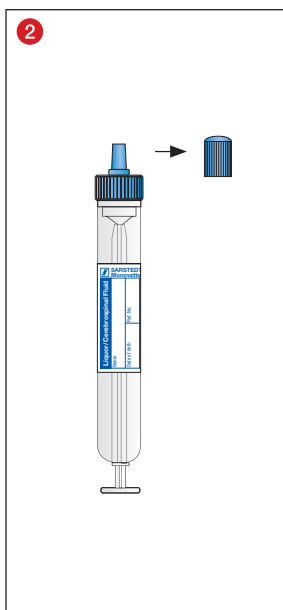
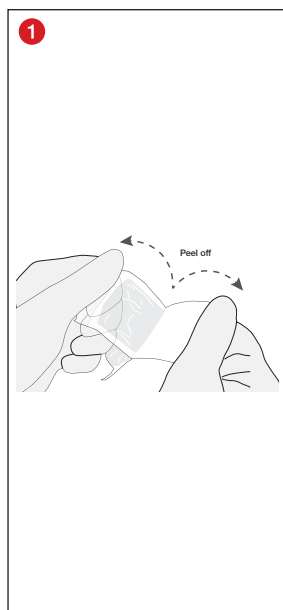
Equipment required for specimen collection

1. Gloves, gown, goggles or other suitable protective wear to protect against CSF-borne pathogens or potentially infectious materials.
2. Required number of CSF Monovettes.
3. Block rack or other suitable device for placing the filled CSF Monovettes.
4. Disposal container for hazardous biological waste.

NOTE: Check whether specific pre-analytical handling processes are necessary according to the test manufacturer and follow these steps.

Handling of the CSF Monovette®

- 1 To place the sterile CSF Monovette®, open the blister pack using the peel-off technique over a sterile cloth. The transparent film must be facing upwards.
- 2 Hold the CSF Monovette® upright with the tip pointing upwards. Loosen the stopper on the Luer screw cap and set it aside for later.
- 3 Remove the CSF sample from the primary receptacle by slowly pulling the plunger backwards until the desired sample volume is reached.
- 4 Take the CSF Monovette® out of the liquid and hold it upright with the tip pointing upwards.
- 5 Pull the plunger downwards to the stop.
- 6 Close the CSF Monovette® securely with the stopper that was previously set aside.
- 7 Break off the plunger rod.
- 8 Place the CSF Monovette® in an upright position according to the intended storage conditions for the specimen material.

Freezing/thawing

The following general recommendations for freezing apply to CSF Monovettes:

Freezing below 0 °C

- Check beforehand whether freezing will adversely affect the CSF sample or analysis.

NOTE: For the stability of the parameters, consult the instructions for use of the test reagents / analysis device manufacturers.

- The strength values of plastics are reduced in the temperature range below 0 °C. Mechanical stress should therefore be avoided.
- The conditions for freezing must be selected so that the contents of the CSF Monovette® freeze evenly or from bottom to top. The CSF Monovette® should have enough room in a rack or storage box to allow it to expand. Holders made of Styrofoam or metal are unsuitable as they can lead to expansion tears.

Freezing at -20 °C

- Cool the CSF Monovette® in an upright position for a period of 45-60 min. from room temperature to +4 °C before it can be frozen at -20 °C.

Freezing below -20 °C

- Deep-freezing to below -20 °C has not been tested by the manufacturer. Due to the number of possible influencing factors, it is recommended to conduct freezing tests under routine laboratory conditions.

Thawing

- Allow the CSF Monovette® to thaw at least 45 min. at room temperature while standing upright. Mechanical stress must also be prevented in this step. Thawing too quickly can impair the results of the analysis.

Centrifugation

WARNING! SARSTEDT CSF Monovettes are designed for maximum 3000 x g (RCF – relative centrifugal force – g force).

Centrifuge inserts are to be selected according to the size of the CSF Monovettes used. The relative centrifugal force has the following ratio to the selected rpm:

$$\text{RCF (g force)} = 11.2 \times r \times (\text{rpm}/1000)^2$$

"RCF": "Relative centrifugal force"

"rpm": "Revolutions per minute",

"r" [in cm]: "Radius of rotation from the middle of the centrifuge to the bottom of the CSF Monovette®."

Use only suitable tube holders or inserts. Centrifuging CSF Monovettes with cracks or centrifuging at an excessive centrifugal force can lead to breakage of the CSF Monovettes, which can release potentially hazardous substances.

CSF Monovettes should be centrifuged in accordance with the centrifugation conditions listed below. If other conditions are used, they must be validated by the user themselves.

It must be ensured that the CSF Monovettes fit well in the centrifuge inserts. CSF Monovettes that protrude above the insert can get caught in the centrifuge head and be broken. Ensure that the centrifuge is filled evenly. Please refer to the centrifuge instructions for use for further information.

CAUTION! Do not remove broken CSF Monovettes by hand.

Refer to the centrifuge instructions for use for information on disinfecting the centrifuge.

Centrifugation recommendation:

The recommended RCF force (g force) for the centrifugation of CSF samples with precipitates is 2,000 x g for 10 minutes.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled CSF Monovettes must be disposed of in suitable containers for hazardous biological waste.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Product-specific standards and guidelines as amended/updated

DIN EN 14254: In vitro diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Store away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture
	Sterilisation by irradiation
	Single sterile barrier system with external protective packaging
	Do not use if packaging is damaged
	Do not resterilise

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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