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Intended use

The glass tube for the Coombs test is used for collecting and processing blood samples for *in vitro* diagnostic tests, the direct and indirect Coombs test. The glass tube for the Coombs test is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The glass tube for the Coombs test consists of transparent glass with a round base. The glass tube for the Coombs test is a single-use product and was developed for the Coombs test. The specimen material in the glass tube for the Coombs test can be sent for automatic or manual specimen processing. The specifications of the device manufacturer must be observed.

Safety information and warnings

1. Do not use a damaged glass tube for the Coombs test.
2. General precautions: Use gloves and other general personal protective equipment to protect yourself from blood and possible exposure to pathogens transmitted by blood.
3. Handle all biological specimens according to the guidelines and procedures in your facility.
In case of direct contact with biological specimens, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases.
Always follow the safety guidelines and procedures of your facility.
4. Dispose of the used and contaminated glass tube for the Coombs test in suitable containers for hazardous biological waste.
5. The product is intended for single use.
6. Do not use the product after the expiry date. The expiry date of the glass tube for the Coombs test is the last day of the month and year indicated.

Storage

Store the product at room temperature.

Handling

READ ALL OF THIS DOCUMENT BEFORE YOU START USING THE TUBE.

Material required for the Coombs test:

1. Suitable protective clothing according to the hygiene guidelines of your facility.
2. All material for conducting the test in accordance with the instructions for use of the test manufacturer and the Standard Operating Procedure (SOP) of your facility.
3. Disposal container for hazardous biological waste.

Handling the glass tube for the Coombs test

1. Open the packaging of the glass tube for the Coombs test and take it out.
2. Use the glass tube for the Coombs test in accordance with the instructions for use of the test manufacturer and the Standard Operating Procedure (SOP) of your facility.
3. The glass tube for the Coombs test can be used for either manual or automated analysis.
4. For use in devices, follow the respective device manufacturer's instructions.

Centrifugation

SARSTEDT glass tube for the Coombs test is designed for a maximum of 3,000 x g for 10 minutes (RCF: relative centrifugal force – g-force). The relative centrifugal force has the following ratio to the selected rpm:

$$\text{RCF (g-force)} = 11.2 \times r \times (\text{rpm}/1000)^2$$

"RCF": "Relative centrifugal force",

"rpm": "Revolutions per minute" (min⁻¹),

"r" [in cm]: "Centrifuge radius from the centre of the centrifuge to the bottom" of the glass tube for the Coombs test.

Use only suitable centrifuge inserts.

WARNING!

Centrifuging the glass tube for the Coombs test with cracks or centrifuging at an excessive centrifugal force can lead to breakage of the glass tube for the Coombs test, which can release potentially hazardous substances.

The glass tube for the Coombs test should be centrifuged according to the centrifugation conditions listed and not exceeded. If other conditions are used, they must be validated by the user themselves. It must be ensured that the glass tube for the Coombs test fits well in the centrifuge inserts. Glass tubes for the Coombs test that protrude beyond the insert can catch on the centrifuge head and be broken. Ensure that the centrifuge is filled evenly. Refer to the instructions for use of the centrifuge.

CAUTION! Do not remove broken glass tubes for the Coombs test by hand.

Refer to the centrifuge instructions for use for information on disinfecting the centrifuge.

Centrifugation recommendation:

The recommended RCF force (g-force) for the Coombs test is 1000 x g for 30 seconds.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated glass tubes for the Coombs test must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Store away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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