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CLSI* GP41 "Collection of Diagnostic Venous Blood Specimens", Approved Standard.

*CLSI (Clinical and Laboratory Standards Institute)

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Intended use

The adapter cap is used to securely adapt an S-Monovette® to a vacuum needle holder for venous blood collection. The product is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The adapter cap is made of plastic and was developed especially for connecting an S-Monovette® to a vacuum needle holder. The adapter cap fits exactly on the screw cap of an S-Monovette®. The adapter cap is a disposable product.

Note: The adapter cap is suitable for vacuum needle holders with a diameter greater than 17.9 mm.

Safety information and warnings

1. General precautions: Use gloves and other general personal protective equipment to protect yourself from blood and possible exposure to potentially infectious specimen material and transmissible pathogens.
2. Handle all biological samples and sharp blood collecting utensils (needles) in accordance with the guidelines and procedures in your facility. In case of direct contact with biological specimens or of a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases. Always follow the safety guidelines and procedures of your facility.
3. The product is intended for single use. Dispose of all sharp objects (needles) for blood collection in suitable containers for hazardous biological waste.
4. If blood is collected through an intravenous (IV) line, it must be ensured that the line is properly flushed in accordance with facility guidelines (=cleaned of the IV solution) before beginning blood collection. Properly flushing the line prevents faulty analysis results.
5. Dispose of used and contaminated needles and blood collection accessories such as the adapter cap in suitable disposal containers for biological hazardous substances.
6. Do not use the product after the expiry date. The expiry date of the product is the last day of the month and year indicated.

Storage

Store the product at room temperature.

Limitations

The adapter cap is suitable for vacuum needle holders with a diameter greater than 17.9 mm.

Only use the S-Monovette® with adapter cap with the vacuum technique.

Collecting and handling specimens

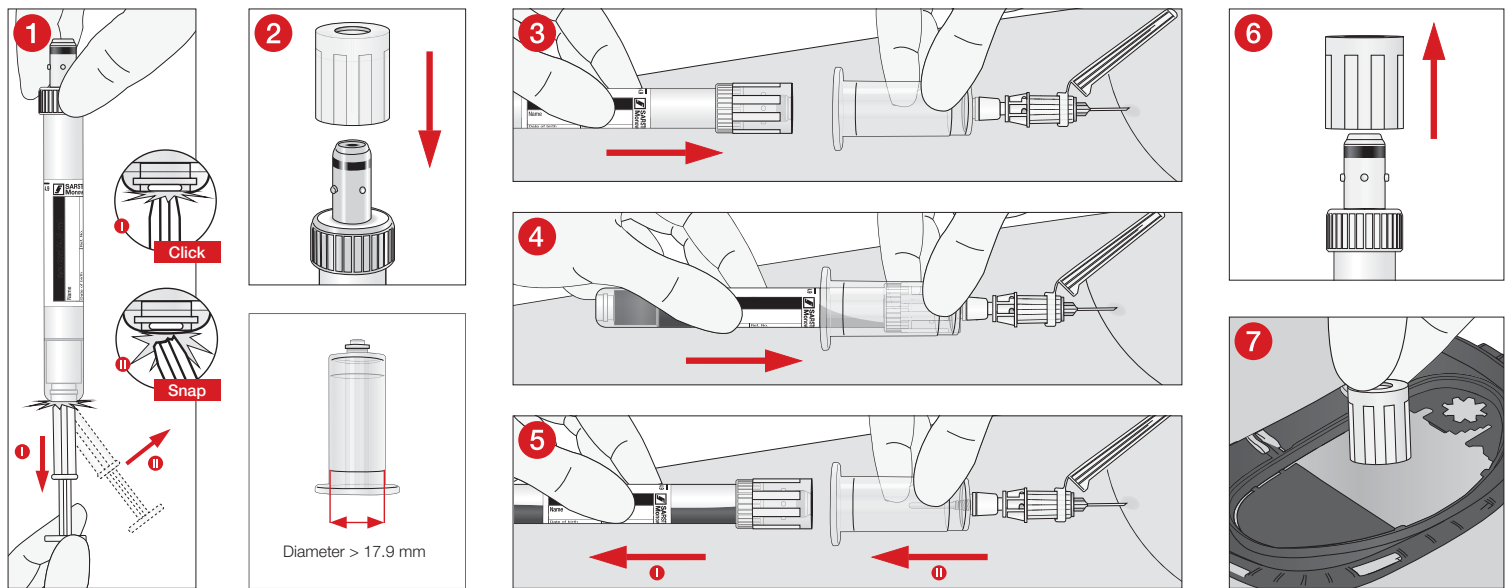
READ THIS DOCUMENT COMPLETELY.

Equipment required for specimen collection

1. All necessary S-Monovette® blood collection systems.
2. Vacuum needle holder, suitable (safety) needles and adapter cap.
3. Gloves, gown, goggles or other suitable protective gear to protect against bloodborne pathogens or potentially infectious materials.
4. Labels for patient identification.
5. Disinfectant to clean the collection site (follow the guidelines of the facility for sterile specimen collection for preparing the collection site). Do not use any alcohol-based cleaning materials if the specimens are to be used for a blood alcohol test.
6. Dry, clean, disposable swabs.
7. A tourniquet.
8. Plaster or dressing.
9. Sharps disposal container for the safe disposal of used material.

Blood collection with the adapter cap

FOLLOW THE INSTRUCTIONS FOR USE OF THE S-MONOVETTE®: USE WITH THE VACUUM TECHNIQUE.

- 1 Before collecting blood with the vacuum technique, prepare the S-Monovette® (vacuum is freshly generated): use the plunger rod to pull the plunger to the bottom of the S-Monovette® and allow it to snap into place audibly **I**. Then the plunger rod is broken off **II**.
- 2 Put the adapter cap on the screw cap of the S-Monovette® to be used and ensure that the vacuum needle holder has a diameter greater than 17.9 mm.
- 3 Puncture the vein with the (safety) needle with the vacuum needle holder attached (according to the manufacturer's instructions for use). Then push the prepared S-Monovette® into the vacuum needle holder until the membrane in the screw cap is pierced.
- 4 Wait until the blood stops flowing to allow correct filling.
- 5 The S-Monovette®, including the adapter cap, is disconnected from the (safety) needle by pulling it out of the vacuum needle holder **I**. The (safety) needle with the attached vacuum needle holder remains in the vein. For multiple blood draws, repeat steps **1** – **5**. Disconnect the S-Monovette®, including the adapter cap, from the vacuum needle holder **I**, then pull the (safety) needle with the vacuum needle holder out of the vein **II** and if it is a safety needle, activate the needle guard according to the manufacturer's instructions for use. Invert each S-Monovette® once immediately after disconnecting it from the vacuum needle holder, and after completing blood collection, carefully invert all specimens with anticoagulants several times.
 1. Press a clean, dry swab against the puncture site until bleeding stops.
 2. As soon as coagulation has set in, you may apply a dressing.
 3. Dispose of the (safety) needle with attached vacuum needle holder in a suitable container for hazardous biological waste.
- 6 Take the adapter cap off the screw cap of the S-Monovette®.
- 7 Dispose of it in a suitable container for hazardous biological waste.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled blood collection systems must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Product-specific standards and guidelines as amended/updated

CLSI* GP41 "Collection of Diagnostic Venous Blood Specimens", Approved Standard.
 *CLSI (Clinical and Laboratory Standards Institute)

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE mark
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Keep away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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