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SARSTEDT Formalin System

DE	Gebrauchshinweis – SARSTEDT Formalin System	2 - 5
EN	Instructions for Use – SARSTEDT Formalin System	6 - 9
BG	Инструкции за употреба – SARSTEDT Formalin System	10 - 13
CS	Návod k obsluze – Formalinový systém SARSTEDT	14 - 17
DA	Bruksanvisning – SARSTEDT Formalin System	18 - 21
EL	Οδηγίες χρήσεως – Σύστημα φορμαλίνης SARSTEDT	22 - 25
ES	Instrucciones de uso – Sistema Formalín SARSTEDT	26 - 29
ET	Kasutusjuhend – SARSTEDT formalinisisüsteem	30 - 33
FR	Mode d'emploi – Système Formol SARSTEDT	34 - 37
HR	Uputa za upotrebu – SARSTEDT Sustav s formalinom	38 - 41
HU	Használati utasítás – SARSTEDT formalin rendszer	42 - 45
IT	Istruzioni d'uso – Sistema alla formalina SARSTEDT	46 - 49
KO	사용 설명서 – SARSTEDT 포르말린 시스템	50 - 53
LT	Naudojimo instrukcijos – „SARSTEDT Formalin System“	54 - 57
LV	Lietošanas norādes – SARSTEDT formalīna sistēma	58 - 61
NL	Gebruiksaanwijzing – SARSTEDT Formalinesysteem	62 - 65
NO	Bruksanvisning – SARSTEDT-formalinsystem	66 - 69
PL	Instrukcja obsługi – system z formaliną SARSTEDT	70 - 73
PT	Instruções de utilização – Sistema de formalina SARSTEDT	74 - 77
RO	Instrucțiuni de utilizare – Sistem cu formol SARSTEDT	78 - 81
RU	Инструкция по применению – формалиновая система SARSTEDT	82 - 85
SK	Návod na Použitie – Formalinový systém SARSTEDT	86 - 89
SL	Navodila za uporabo – Sistem formalin SARSTEDT	90 - 93
SV	Bruksanvisning – SARSTEDT Formalinsystem	94 - 97
TH	คำแนะนำในการใช้งาน – ระบบฟอร์มาลีนของ SARSTEDT	98 - 101
TR	Kullanım için talimatlar – SARSTEDT Formalin System	102 - 105
ZH	使用说明 – 莎斯特 (SARSTEDT) 福尔马林系统	106 - 109

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Intended use

The formalin system is used for the collection of tissue specimens, adding formalin to the specimen in a closed system and short-term storage. The products are intended for use in a professional environment by qualified medical or laboratory personnel.

Product description

The formalin system is used to collect and fix human tissue specimens and consists of the following components:

- A formalin bag with 4 % buffered formaldehyde or alternatively 4 % buffered formaldehyde with additional 0.1 % erythrosine
- One of the two versions of the S-Monovette® (9 ml or 25 ml).

The components are available as a set or individually. Additionally a wall holder is available as an accessory for the stable storage of the formalin bag. In one version of the formalin system, 0.1 % of the colourant erythrosine (REF 51.1703.001) is added to the formalin solution to dye the tissue specimen. The S-Monovettes are packed in bags, while the 9 ml S-Monovette® (REF 09.1704.021) is a sterile version packed individually in blister packaging. The sterility of the S-Monovette® is not necessary for fixing the specimen, but is an adaptation to a probable sterile work environment.

Type	Art. no.	Description
A	51.1703	Formalin system with 450 ml formaldehyde solution
A	51.1703.001	Formalin system with 450 ml formaldehyde solution, dyed red
A & B	51.1703.009	Formalin system with 450 ml formaldehyde solution and 100 S-Monovette® 9 ml specimen tubes
A & C	51.1703.025	Formalin system with 450 ml formaldehyde solution and 50 S-Monovette® 25 ml specimen tubes
B	09.1704.xxx	S-Monovette® 9 ml, 92 x 16 mm
C	09.1705.xxx	S-Monovette® 25 ml, 97 x 25 mm

Safety information and warnings

NOTE FOR STERILE VERSIONS: DO NOT USE THE PRODUCT IF THE BLISTER PACKAGING IS DAMAGED.

1. General precautions:
Use gloves and general personal protective equipment to protect yourself from possible exposure to potentially infectious specimen material and transmitted pathogens.
2. Handle all biological specimens and collection utensils according to the guidelines and procedures in your facility. In the event of contact with biological samples, consult a doctor, as this can transmit infectious diseases.
3. Dispose of all sharp objects for specimen collection in appropriate disposal containers.
4. Always follow the safety guidelines and procedures of your facility.
5. Do not use the product after the expiry date. The expiry date of the product is the last day of the month and year indicated.
6. The formalin system is intended for multiple collections. After using all the formalin solution, dispose of the products according to the guidelines of your facility.
7. The S-Monovettes are intended for single use. Dispose of all products in the hazardous biological waste containers.

Hazard statements

H302	Harmful if swallowed.
H317	May cause an allergic skin reaction.
H341	Suspected of causing genetic defects.
H350	May cause cancer.

Precautionary statements

P201	Obtain special instructions before use.
P202	Do not handle until all safety precautions have been read and understood.
P261	Avoid breathing mist/vapours/spray.
P264	Wash hands thoroughly after handling.
P270	Do not eat, drink or smoke when using this product.
P272	Contaminated work clothing should not be allowed out of the workplace.
P280	Wear protective gloves/eye protection.
P301+P312	IF SWALLOWED: Call a POISON CENTER/doctor if you feel unwell.
P330	Rinse mouth.
P302+P352	IF ON SKIN: Wash with plenty of water.
P321	Specific treatment (see information on this label).
P362+P364	Take off contaminated clothing and wash it before reuse.
P308+P313	IF exposed or concerned: Get medical advice/attention.
P405	Store locked up.
P501	Dispose of contents/container according to the regulations.

Storage

Store the products at room temperature.

Transport

The S-Monovettes together with an ADR/IATA (packaging instruction P650) compliant secondary container, incl. suction insert and type-tested outer packaging, comply with the ADR P650/IATA regulations. Always transport the S-Monovettes to the laboratory as quickly as possible.

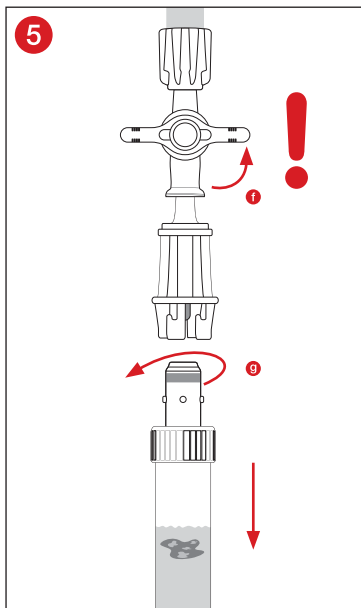
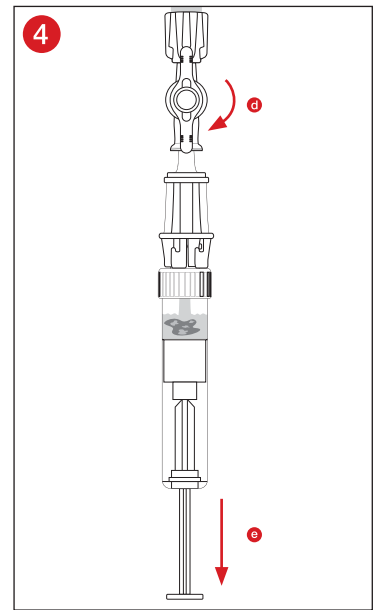
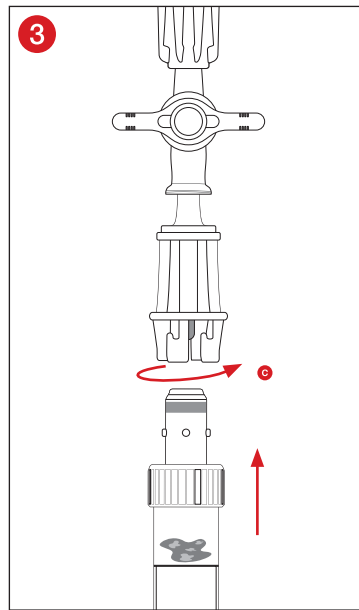
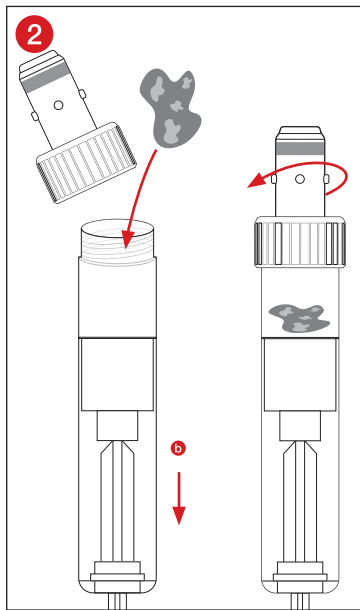
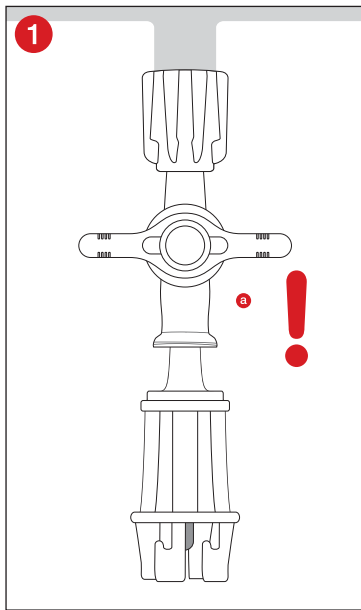
Disposal

- The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
- Disposable gloves prevent the risk of infection.
- Dispose of potentially contaminated material according to the regulations and guidelines of the facility.

All serious incidents associated with this product must be reported to the manufacturer and the national authorities where the user is located.

Handling

- 1 Ensure that the system does not leak and is undamaged.
Hang the system on the wall holder intended for this purpose (REF 95.1706).
Check that the tap is in the closed, horizontal position **a**.
Ensure that the fluid in the bottle is clear and transparent. Dispose of the system if there is any sign of cloudiness.
- 2 Unscrew the cap of the S-Monovette® to place the specimen in it. If the S-Monovette® is not completely filled, pull the plunger of the S-Monovette® down a little **b**. Close the S-Monovette® again with the screw cap.
- 3 Now connect the S-Monovette® to the adapter of the formalin system and secure the connection by twisting the S-Monovette® approx. 2-3 mm to the right **c**.
- 4 Open the tap by twisting it 90° **d**.
You can then fill the S-Monovette® with formalin by pulling the plunger down **e**.
Please note that the plunger must be pulled down to the stop and air in the upper section is unavoidable. Do not press the plunger back against the direction of withdrawal. If this should occur, the system must be disposed of.
- 5 Close the system by twisting the tap back 90° **f**.
To detach the S-Monovette® from the system, twist it approx. 2-3 mm to the left **g**.
Check again that the tap of the formalin system is in the closed, horizontal position.
To ship the specimen, break off the protruding plunger rod of the S-Monovette®.

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Store away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

The following also applies to sterile products:

	Sterilisation by irradiation
	Single sterile barrier system with external protective packaging
	Do not use if package is damaged
	Do not resterilise

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