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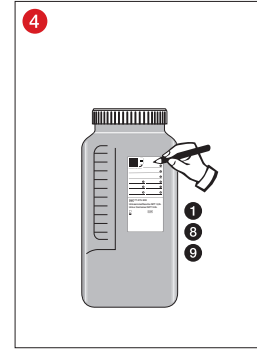
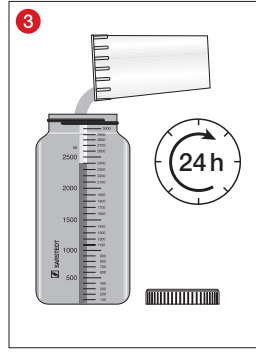
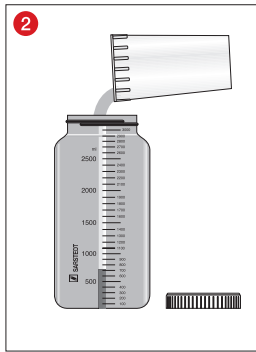
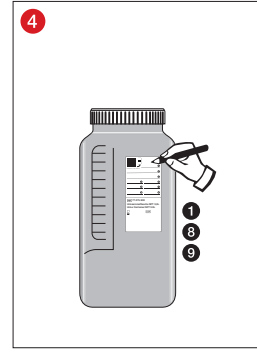
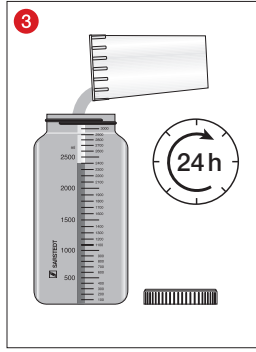
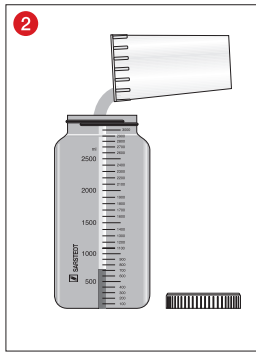
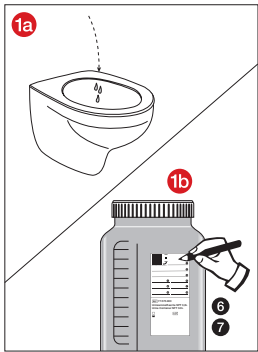
Date collection began ⑥ Time ⑦

Date collection ended ⑧ Time ⑨

Perform the 24-hour urine collection as directed by your doctor. If you have not been provided with any specific instructions start the 24-hour urine collection as follows.

- ① Empty your bladder into the toilet in the morning when you get up. (Fig. 1a). Note the date and time under ⑥ and ⑦ on the container label (Fig. 1b).
- ② Collect your next urine in the NFT urine collection container. You may use a small collection cup, but one that is sufficiently large, and transfer all the urine to the collection container. Collect all further urine, including when defecating, and put it in the NFT urine collection container. Close the NFT urine collection container well after any additions.
- ③ Go to the toilet the next day at the time noted on the day before and collect this urine. Collection is then complete.
- ④ Read the collected quantity on the graduated scale on the NFT urine collection container. Enter this number under ① on the container label and add the date and time of the most recent addition under ⑧ and ⑨.
- ⑤ Enter complete patient data under ② – ⑤. Bring the filled NFT urine collection container to your doctor or proceed as instructed by your doctor.

Note: Please store the urine collection container NFT away from heat sources and sunlight for the entire collection period.

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