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SARSTEDT Salivette® VD

DE	Gebrauchshinweis – SARSTEDT Salivette® VD	2 - 4
EN	Instructions for use – SARSTEDT Salivette® VD	5 - 7
BG	Инструкции за употреба – SARSTEDT Salivette® VD	8 - 10
CS	Návod k obsluze – SARSTEDT Salivette® VD	11 - 13
DA	Brugsanvisning – SARSTEDT Salivette® VD	14 - 16
EL	Οδηγίες χρήσεως – SARSTEDT Salivette® VD	17 - 19
ES	Instrucciones de uso – SARSTEDT Salivette® VD	20 - 22
ET	Kasutusjuhend – SARSTEDT Salivette® VD	23 - 25
FR	Mode d'emploi – SARSTEDT Salivette® VD	26 - 28
HR	Uputa za upotrebu – SARSTEDT Salivette® VD	29 - 31
HU	Használati utasítás – SARSTEDT Salivette® VD	32 - 34
IT	Istruzioni d'uso – SARSTEDT Salivette® VD	35 - 37
KO	사용 설명서 – SARSTEDT Salivette® VD	38 - 40
LT	Naudojimo instrukcijos – SARSTEDT Salivette® VD	41 - 43
LV	Lietošanas norādes – SARSTEDT Salivette® VD	44 - 46
NL	Gebruiksaanwijzing – SARSTEDT Salivette® VD	47 - 49
NO	Bruksanvisning – SARSTEDT Salivette® VD	50 - 52
PL	Instrukcja obsługi – SARSTEDT Salivette® VD	53 - 55
PT	Instruções de utilização – SARSTEDT Salivette® VD	56 - 58
RO	Instrucțiuni de utilizare – SARSTEDT Salivette® VD	59 - 61
RU	Инструкция по применению – SARSTEDT Salivette® VD	62 - 64
SK	Návod na Použitie – SARSTEDT Salivette® VD	65 - 67
SL	Navodila za uporabo – SARSTEDT Salivette® VD	68 - 70
SV	Bruksanvisning – SARSTEDT Salivette® VD	71 - 73
TH	คำแนะนำในการใช้งาน – SARSTEDT Salivette® VD	74 - 76
TR	Kullanım için talimatlar – SARSTEDT Salivette® VD	77 - 79
ZH	使用说明 – SARSTEDT Salivette® VD	80 - 82

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Literatur

Hoch et al Diagnostics 11(10):1707 2021, Feasibility and Diagnostic Accuracy of Saliva-Based SARS-CoV-2 Screening in Educational Settings and Children Aged <12 Years, DOI: 10.3390/diagnostics11101797
 Wyllie et al NEJM 383:13 2020, Saliva or Nasopharyngeal Swab Specimens for Detection of SARS-CoV-2, DOI: 10.1056/NEJMc2016359

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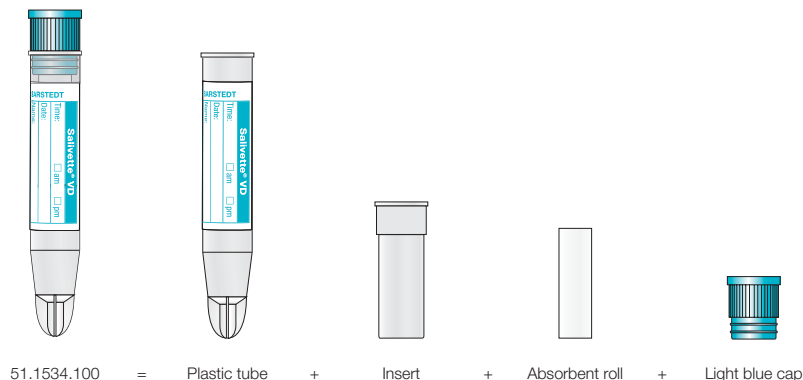
Intended use

The Salivette® VD is a specimen tube for saliva samples and is used for the collection, transport, processing and storage of saliva for screening of virus material, e.g. SARS-CoV-2 nucleic acids, in *in vitro* diagnostic tests. The product is intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The Salivette® VD consists of a transparent conical bottom plastic tube, an insert, an absorbent cotton roll and a light blue cap. The absorbent cotton roll is a natural product. Discolouration and darker inclusions or areas are natural and unavoidable.

Safety information and warnings

1. **The product must not be used in children under 3 years of age or in patients at increased risk of choking.**
2. General precautions: Use gloves and other general personal protective equipment to protect yourself against possible exposure to potentially infectious sample material. Always follow the safety guidelines and procedures of your facility.
3. The saliva collected and processed with the Salivette® VD is not intended to be returned to the human body.
4. Do not use Salivette® VD after the expiry date. The expiry date of Salivette® VD is the last day of the month and year as indicated.

Storage

Store Salivette® VD at room temperature before use.

Transport

The product can be transported manually in-house or with an ADR-compliant (packaging instruction P650) secondary container. The ADR-compliant (packaging instruction P650) shipping container, Art. no. 78.898, with absorbent insert plus screw cap, Art. no. 65.679, and shipping box/bag is recommended for Salivette® VD specimen transport.

For the patient:

To return the saliva sample in the tightly closed Salivette® VD, follow the doctor's instructions and use the secondary container provided by the doctor and the shipping box/bag. If it is not returned immediately, store this Salivette® VD in the refrigerator.

Limitations

The saliva sample should be taken at least 60 minutes after a meal (liquid/solid food intake), brushing teeth or taking oral medication, in order to avoid the saliva being contaminated with interfering substances.

The storage duration and temperature depend on the shelf life for the parameters to be tested. Therefore, and since the onset of bacterial growth in saliva can be expected after just a few hours at room temperature, it is recommended that the Salivette® VD be sealed tightly after sampling and either centrifuged and analysed within 4 hours or placed immediately in a refrigerator until further processing.

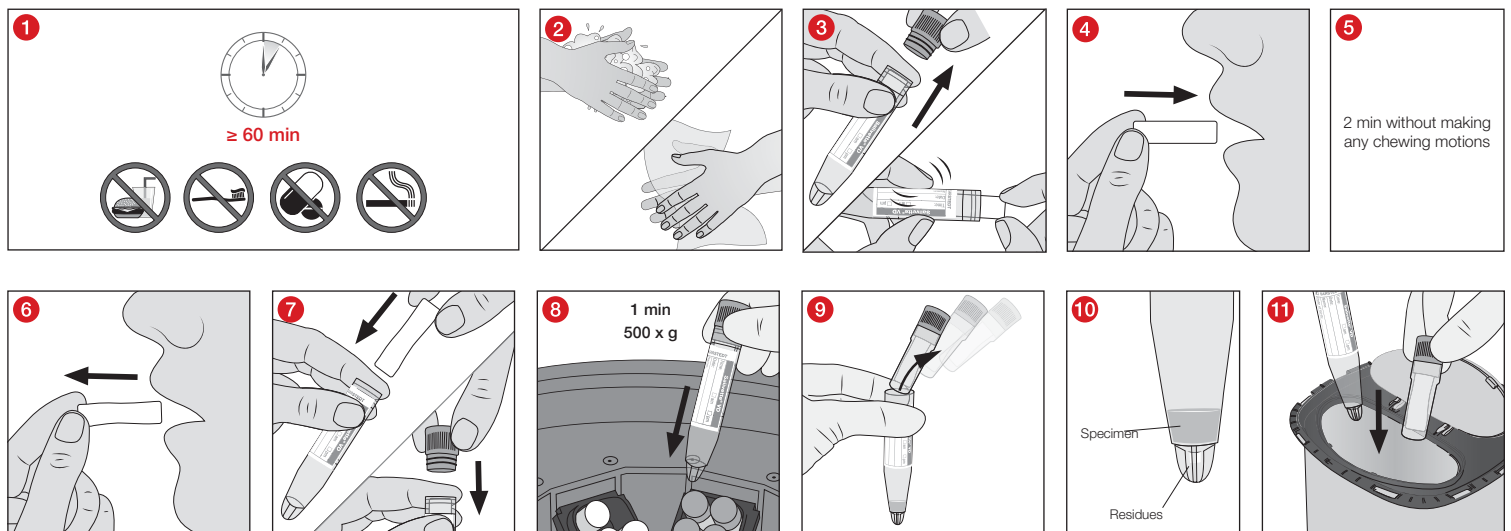
Specimen collection and handling

READ ALL OF THIS DOCUMENT BEFORE STARTING TO COLLECT SALIVA.

Material required to collect a specimen:

1. Gloves, gown, goggles or other suitable protective gear to protect against pathogens transmitted via saliva or potentially infectious materials.
2. Number of Salivette® VD tubes as requested by the doctor.
3. Fill out information on sample identification (on label).

Sample collection

- 1 The saliva sample should be taken at least 60 minutes after a meal (liquid/solid food intake), brushing teeth, smoking or taking oral medication, in order to avoid the saliva being contaminated with interfering substances.
 - 2 Immediately before collecting the saliva, wash and dry hands.
 - 3 With two fingers, remove the absorbent roll at the far end from the opened Salivette® VD and
 - 4 place it in the mouth.
 - 5 The absorbent roll should be kept in the mouth (e.g. in the cheek) for 2 minutes without chewing. The amount of saliva collected should be at least 500 µl. If an extremely small amount of saliva is produced, leave the absorbent roll in the mouth longer.
 - 6 With two fingers, take the absorbent roll out of the mouth by the far end and
 - 7 place it immediately in the Salivette® VD and seal with the light blue cap.
For the patient: To return the saliva sample in the tightly closed Salivette® VD, follow the doctor's instructions and use the secondary container provided by the doctor and the shipping box/bag. If it is not returned immediately, store this Salivette® VD in the refrigerator.
- For doctors and laboratories:
- 8 Because of the onset of bacterial growth in saliva after a few hours at room temperature, it is recommended to either centrifuge the firmly closed Salivette® VD within 4 hours after specimen collection at 500 x g for 1 minute and analyse it or to store it immediately in the refrigerator until further processing.
 - 9 After centrifugation, take the insert including cap out of the conical bottom plastic tube.
 - 10 The pure saliva for further analysis is found in the conical bottom of the plastic tube above the residues that are located in the reservoir of the conical bottom.
 - 11 After analysis, dispose of the conical bottom plastic tube and the insert in a disposal container.

Disposal – for doctors and laboratories

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled Salivette® VD tubes must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Literature

Hoch et al. *Diagnostics* 11(10):1707 2021, Feasibility and Diagnostic Accuracy of Saliva-Based SARS-CoV-2 Screening in Educational Settings and Children Aged <12 Years, DOI: 10.3390/diagnostics11101797
 Wyllie et al. *NEJM* 383:13 2020, Saliva or Nasopharyngeal Swab Specimens for Detection of SARS-CoV-2, DOI: 10.1056/NEJMc2016359

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	<i>In vitro</i> diagnostic device
	Follow the instructions for use
	If reused: Risk of contamination
	Store away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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