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SARSTEDT Urinsammelflaschen

DE	Gebrauchshinweis – SARSTEDT Urinsammelflaschen	2 - 5
EN	Instructions for Use – SARSTEDT Urine Collection Containers	6 - 9
BG	Инструкции за употреба – Контейнери за вземане на урина SARSTEDT	10 - 13
CS	Návod k obsluze – SARSTEDT Sběrné lahve na moč	14 - 17
DA	Brugsanvisning – SARSTEDT urindunke	18 - 21
EL	Οδηγίες χρήσεως – Δοχεία συλλογής ούρων SARSTEDT	22 - 25
ES	Instrucciones de uso – SARSTEDT Contenedores para recogida de orina	26 - 29
ET	Kasutusjuhend – SARSTEDTI uriinikogumispudelid	30 - 33
FR	Mode d'emploi – Flacon à urine SARSTEDT	34 - 37
HR	Uputa za upotrebu – SARSTEDT boce za prikupljanje urina	38 - 41
HU	Használati utasítás – SARSTEDT vizeletgyűjtő palackok	42 - 45
IT	Istruzioni d'uso – Contenitori per la raccolta delle urine SARSTEDT	46 - 49
KO	사용 설명서 – SARSTEDT 소변 수집병	50 - 53
LT	Naudojimo instrukcijos – SARSTEDT šlapimo surinkimo buteliai	54 - 57
LV	Lietošanas norādes – SARSTEDT urīna savākšanas tvertnes	58 - 61
NL	Gebruiksaanwijzing – SARSTEDT Urineverzamel flessen	62 - 65
NO	Bruksanvisning – SARSTEDT urindunk	66 - 69
PL	Instrukcja obsługi – butelki zbiorcze na moc SARSTEDT	70 - 73
PT	Instruções de utilização – Frascos para colheita de urina SARSTEDT	74 - 77
RO	Instrucțiuni de utilizare – Flacoane de colectare a urinei SARSTEDT	78 - 81
RU	Инструкция по применению – Контейнеры для сбора мочи SARSTEDT	82 - 85
SK	Návod na Použitie – Zberné fľaše na moč SARSTEDT	86 - 89
SL	Navodila za uporabo – posode za zbiranje urina SARSTEDT	90 - 93
SV	Bruksanvisning – SARSTEDT urinuppsamlingsflaskor	94 - 97
TR	Kullanım için talimatlar – SARSTEDT İdrar Toplama Kaplan	98 - 101
ZH	使用说明 – SARSTEDT 集尿瓶	102 - 105

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DIN EN 14254: *In-vitro* diagnostic medical devices - Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

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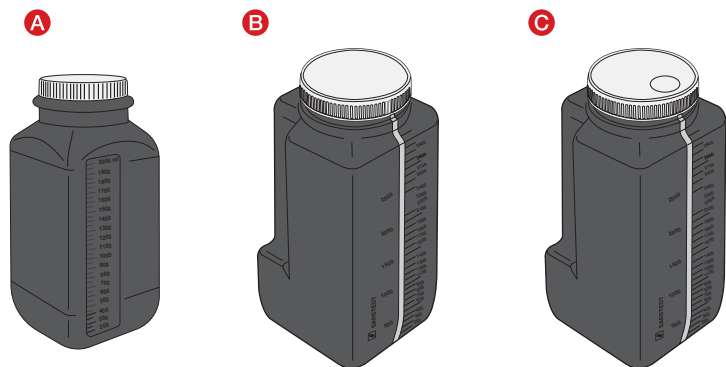
Intended use

The urine collection containers are used as a specimen collection vessel for collecting, transporting and storing urine for *in vitro* diagnostics. The products are intended for use by patients (specimen collection) under the instruction of qualified medical personnel and in a professional environment by qualified medical and laboratory personnel.

Product description

The urine collection containers consist of a plastic vessel with a wide opening diameter and suitable screw cap. They are available for different filling volumes and depending on the version, have light protection, graduation marks, viewing window, recessed holders, a self-adhesive patient label and an integrated transfer unit for filling a V-Monovette® Urine.

Product overview

Urine collection containers

Type	Description
A	Urine collection container, 2 l, brown, light protection, graduation marks, enclosed self-adhesive patient label, label language DE/EN, FR/NL
B	Urine collection container, 3 l, brown, light protection, graduation marks, viewing window, recessed holders, enclosed self-adhesive patient label, label language depending on the order number (DE, EN, FR)
C	Urine collection container with integrated transfer unit, 3 l, brown, light protection, graduation marks, viewing window, recessed holders, enclosed self-adhesive patient label, label language depending on the order number (DE, EN, FR)

Urine collection container accessories

Self-adhesive label for urine collection container A, B and C

Order no.	Dimensions	Version	Packaging
84.1402	150x55 mm	Label in German	300/inner carton (10x30/bag)
84.1402.001	150x55 mm	Label in English	300/inner carton (10x30/bag)
84.1402.002	150x55 mm	Label in French	300/inner carton (10x30/bag)
84.1402.003	150x55 mm	Label in Dutch	300/inner carton (10x30/bag)

Safety information and warnings

1. General precautions: Wear gloves and other general personal protective equipment to protect against possible exposure to potentially infectious specimen material and transmitted pathogens.
2. Handle all biological specimens and collection utensils according to the guidelines and procedures in your institution. In case of direct contact with biological specimens or a needlestick injury, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases. Always follow the safety guidelines and procedures of your facility.
3. Urine collection container with an integrated transfer unit: The safety label should remain over the transfer unit for the entire time urine is collected. To prevent needlestick injuries, never put fingers in the transfer unit.
4. Always store the filled urine collection container closed, upright, and out of the reach of children.
5. Collection container and cap are intended for single use only. Dispose of all products in the hazardous biological waste containers.
6. Do not use the product after the expiry date. The expiry date of a urine collection container refers to the last day of the month and year indicated.

Storage

Store the products at room temperature.

Transportation

The products can be transported together with an ADR (Packing instruction P650) conform secondary vessel, for example B 17 transport case, Art. no. 95.1717.

Limitations

1. The appropriate storage and stabilisation of the urine during the 24-hour collection period depend on the parameters to be examined. The laboratory should provide precise information on this.
2. The suitability of the urine collection containers for the analysis of trace elements/metals has not been tested.

Collecting and handling specimens

READ ALL OF THIS DOCUMENT BEFORE YOU START URINE COLLECTION.

Equipment required for specimen collection

1. If needed, collection cup for urine portions, for example 500 ml cup, Art. no. 75.1356.

Collecting urine over 24 hours

Empty your bladder in the morning when you get up. Note the date and time. Use the patient label for this if your doctor has given it to you and complete the patient data.

1. Collect the next urine in the collection container. You may use a small collection cup, but one that is sufficiently large and transfer all the urine to the collection container.
2. Close the collection container well.
3. Collect all further urine, even during bowel movements, and put it in the collection container.
4. Close the collection container well every time and swirl it back and forth. Make sure to store it as recommended by your doctor.
5. Go to the toilet the next day at the time noted on the day before, collect this urine and add it to the collection container. The collection phase is then complete.
6. Bring the filled collection container to your doctor.

Procedure for transferring the urine from a collection container with integrated transfer unit to a V-Monovette® Urine

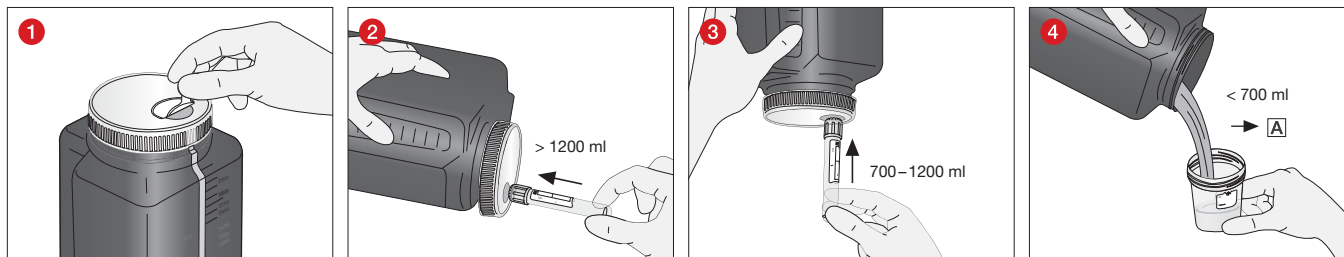
Material required for urine transfer that is not included:

1. Gloves, gown, goggles or other suitable protective wear to protect against potentially infectious materials.
2. Required number of V-Monovettes Urine.
3. Transfer unit (Art. no.: 11.1240) and urine cup if the collection volume is < 700 ml.
4. Sharps disposal container for disposal of used urine transfer units.
5. Disposal container for hazardous biological waste.

Procedure for transferring the urine from a collection container with integrated transfer unit (Type D) to a V-Monovette® Urine:


A: Collection volume > 700 ml

1. Prepare the urine specimen for removal by carefully swirling the urine collection container back and forth.
2. Remove the safety label from the top of the urine collection container to allow access to the integrated transfer unit. Stick the safety label back on the opening after removing the urine, to reseal it. **NOTE:** The minimum filling volume required to fill a V-Monovette® Urine depends on the position of the collection container (Fig. 1).
3. Select a position of the collection container appropriate for the filling volume (Fig. 2 – 4) and guide the V-Monovette® Urine with the cap first into the integrated transfer unit. Make sure that the needle penetrates the stopper of the V-Monovette® Urine. The V-Monovette® Urine starts filling automatically according to the defined vacuum. If no urine flows or the flow stops before a sufficient fill level is achieved, the following steps are recommended:
 - a. Press the V-Monovette® Urine forward firmly again to make sure that the stopper is completely penetrated.
 - b. Secure the V-Monovette® Urine in the transfer unit during the filling process by pressing against the bottom of the V-Monovette® Urine.
 - c. If the urine still does not flow, remove the V-Monovette® Urine and use a new V-Monovette® Urine.
4. Hold the V-Monovette® Urine in place until no more urine flows into it and only then remove it from the integrated transfer unit.
5. Dispose of the urine collection container in a suitable container for hazardous biological waste.

B: Collection volume < 700 ml:

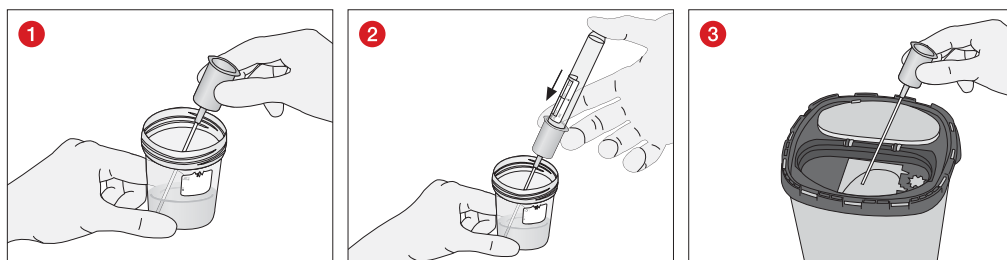1. Prepare the urine specimen for removal by carefully swirling the urine collection container back and forth and transfer a sufficient quantity to a urine cup.
2. Dip the tip of the urine transfer unit into the urine specimen (Fig. 1).
3. Guide the V-Monovette® Urine with the cap pointing downwards into the transfer unit and make sure that the needle penetrates the stopper of the V-Monovette® Urine (Fig. 2). The V-Monovette® Urine starts filling automatically according to the defined vacuum. If no urine flows or the flow stops before a sufficient fill level is achieved, the following steps are recommended:
 - a. Press the V-Monovette® Urine forward firmly again to make sure that the stopper is completely penetrated.
 - b. Secure the V-Monovette® Urine in the transfer unit during the filling process by pressing your thumb against the bottom of the V-Monovette® Urine.
 - c. If the urine still does not flow, remove the V-Monovette® Urine and use a new V-Monovette® Urine.
4. Hold the V-Monovette® Urine in place until no more urine flows into it.
5. Remove the V-Monovette® Urine from the transfer unit and only then take the transfer unit out of the fluid.
6. Dispose of the transfer unit in a suitable container for hazardous biological waste (Fig. 3).
7. Close the urine cup and dispose of it in a container for hazardous biological waste.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled urine collection systems must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Product-specific standards and guidelines as amended / updated

DIN EN 14254: *In vitro* diagnostic medical devices – Single-use receptacles for the collection of specimens, other than blood, from humans
 CLSI* GP16 "Urinalysis Approved Guideline"
 *CLSI (Clinical and Laboratory Standards Institute)

Further literature:

European Urinalysis Guidelines: Scand J. Clin. Lab. Invest 2000; 60: 1–96.

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	For <i>in-vitro</i> diagnostic use
	Follow the instructions for use
	Risk of contamination if reused
	Keep away from sunlight
	Store in a dry place
	Manufacturer
	Country of manufacture

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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