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SARSTEDT Safety-Lanzette

DE	Gebrauchshinweis – SARSTEDT Safety-Lanzette	1 - 4
EN-EU	Instructions for use – SARSTEDT safety lancet	5 - 8
EN-US	Instructions for use – SARSTEDT safety lancet	9 - 12
BG	Инструкции за употреба – SARSTEDT обезопасена ланцета	13 - 16
CS	Návod k obsluze – Bezpečnostní lanceta SARSTEDT	17 - 20
DA	Brugsanvisning – SARSTEDT sikkerhedslancet	21 - 24
EL	Οδηγίες χρήσεως – Σκαφιστήρας ασφαλείας SARSTEDT	25 - 28
ES	Instrucciones de uso – Lanceta de seguridad SARSTEDT	29 - 32
ET	Kasutusjuhend – SARSTEDTI safety lantsett	33 - 36
FR	Mode d'emploi – Lancette de sécurité SARSTEDT	37 - 40
HR	Uputa za upotrebu – SARSTEDT sigurnosna lanceta	41 - 44
HU	Használati utasítás – SARSTEDT biztonságos lán dzsa	45 - 48
IT	Istruzioni d'uso – Lancetta Safety SARSTEDT	49 - 52
KO	사용 설명서 – SARSTEDT 안전 랜셋	53 - 56
LT	Naudojimo instrukcijos – SARSTEDT saugūs lancetai	57 - 60
LV	Lietošanas norādes – SARSTEDT drošības lancete	61 - 64
NL	Gebbruiksaanwijzing – SARSTEDT Safety-lancet	65 - 68
NO	Bruksanvisning – SARSTEDT Safety-lansett	69 - 72
PL	Instrukcja obsługi – Bezpieczny nakłuwacz SARSTEDT	73 - 76
PT	Instruções de utilização – Lanceta de segurança SARSTEDT	77 - 80
RO	Instrucțiuni de utilizare – Lanseță de siguranță SARSTEDT	81 - 84
RU	Инструкция по применению – Безопасный ланцет SARSTEDT	85 - 88
SK	Návod na Použitie – SARSTEDT Bezpečnostná lanceta	89 - 92
SL	Navodila za uporabo – Varna lanceta SARSTEDT	93 - 96
SV	Bruksanvisning – SARSTEDT Safety Lancet	97 - 100
TH	คำแนะนำในการใช้งาน – ไขมีดจิ้มรอยของ SARSTEDT	101 - 104
TR	Kullanım için talimatlar – SARSTEDT Güvenlikli lanset	105 - 108
ZH	使用说明 – SARSTEDT安全采血针	109 - 112

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
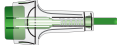
Intended use

The safety lancet with safety function is used for the one-time puncture for capillary blood collection. The products are intended to be used on persons in a professional environment by trained medical personnel.

Product description

The safety lancet is a sterile product for single use. The safety lancet consists of a plastic casing, a disposable needle/blade of stainless steel, a release button in various colours, steel springs and a protective cap. Due to the safety function of the safety lancets, the needle/blade is automatically retracted into the plastic casing after the puncture to prevent needlestick injuries.

Safety lancet product range

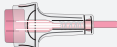Image	Design	Needle size	Penetration depth
	Mini	28 G	1.6 mm
	Normal	21 G	1.8 mm
	Extra	18 G	1.8 mm
	Super	Blade 1.5 mm	1.6 mm
	Neonatal	Blade 1.5 mm	1.2 mm

The different versions of the safety lancet are suitable as needed for collecting various quantities of blood.

The correct version is selected depending on various factors such as skin type, skin blood flow, puncture site and the correct placement of the safety lancet on the skin. In case of doubt, it is recommended to test various safety lancets prior to routine use.

Safety information and warnings

1. Check the protective cap for damage prior to use and, if damaged, do not use it.
2. Follow the directions for handling.
3. Intended for a single use on only one patient. The improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, especially in facilities where multiple patients are tested.
4. General precautions: Use gloves and other general personal protective equipment to protect yourself against blood and possible exposure to pathogens transmitted by blood.
5. Handle all biological specimens and sharp blood collection utensils according to the guidelines and procedures in your facility. Seek a doctor in the event of direct contact with biological samples or a needle-stick injury, as HIV, HCV, HBV and other infectious diseases can be transmitted in this way. Always follow the safety guidelines and procedures of your facility.
6. Dispose of the entire safety lancet after use in a suitable sharps container.
7. Reusing this product could result in infections, injury or death.
8. Do not use the product after the expiry date. The expiry date of the product is the last day of the month and year indicated.

General residual risks of puncture

When using the basic puncture technique for capillary blood collection, there are the following general residual risks according to CLSI GP42:

1. Infections
2. Calcification of the puncture site
3. Scarring
4. Haematomas
5. Bruising

Storage

Store the product at room temperature.

Handling

READ THIS DOCUMENT COMPLETELY BEFORE STARTING THE PUNCTURE.

Materials needed for collecting a specimen:

1. All necessary sample containers, e.g. micro sample tubes, labelled by size and additive as per request.
2. Gloves, lab coat, eye protection or other suitable protective gear to protect against pathogens transmitted by blood or potentially infectious material.
3. Labels for specimen identification.
4. Safety lancet.
5. Disinfectant for cleaning the puncture site (follow the facility's guidelines for preparing the puncture site for blood collection). Do not use alcohol-based cleaning agents if the samples are to be used for blood alcohol tests.
6. Dry, clean, disposable swabs.
7. Plaster or bandage.
8. Sharps disposal container for the safe disposal of used material.

N.B.: Follow the regulations of your facility for the order of collection.

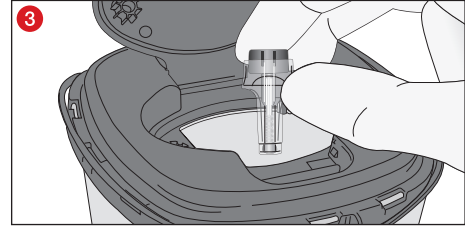
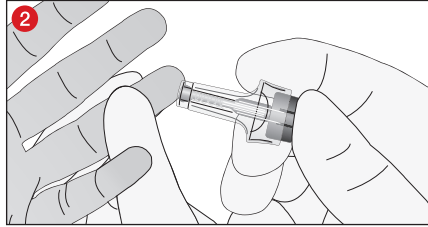
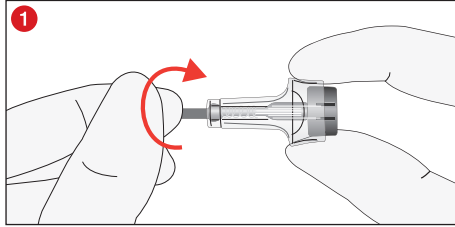
Puncture with the safety lancet

General instructions:

WASH YOUR HANDS BEFORE AND AFTER USING THE SAFETY LANCET.

WEAR GLOVES TO MINIMISE THE RISK OF EXPOSURE DURING CAPILLARY BLOOD COLLECTION.

1. If necessary, select a suitable sample tube or prepare a POC test for the desired sample.
2. Select and disinfect a puncture site for the safety lancet. Follow the facility's guidelines for preparing the puncture site.

- 1 Twist the protective cap off.
- 2 Hold the safety lancet against the selected and disinfected puncture site. Press the release button. The safety lancet automatically retracts into the casing after the puncture.
- 3 Dispose of the safety lancet in a suitable container for sharp objects.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Dispose of the safety lancet in a sharps container.
3. Dispose of potentially contaminated consumables in accordance with the institutional guidelines.

Product-specific standards and guidelines as amended / updated

DIN EN 10088-1: Stainless steels - Part 1: List of stainless steels.

DIN EN ISO 9626: Stainless steel needle tubing for the manufacture of medical devices - Requirements and test methods.

CLSI* GP42 "Collection of Capillary Blood Specimens", Approved Standard.

*CLSI (Clinical and Laboratory Standards Institute)

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Store in a dry place

Store at room temperature

Single sterile barrier system with protective packaging outside

Technical modifications reserved

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

Instructions for Use

SARSTEDT safety lancet

DE	Gebrauchshinweis – SARSTEDT Safety-Lanzette	1 - 4
EN-EU	Instructions for use – SARSTEDT safety lancet	5 - 8
EN-US	Instructions for use – SARSTEDT safety lancet	9 - 12
BG	Инструкции за употреба – SARSTEDT обезопасена ланцета	13 - 16
CS	Návod k obsluze – Bezpečnostní lanceta SARSTEDT	17 - 20
DA	Brugsanvisning – SARSTEDT sikkerhedslancet	21 - 24
EL	Οδηγίες χρήσεως – Σκαφιστήρας ασφαλείας SARSTEDT	25 - 28
ES	Instrucciones de uso – Lanceta Safety SARSTEDT	29 - 32
ET	Kasutusjuhend – SARSTEDTI safety lantsett	33 - 36
FR	Mode d'emploi – Lancette de sécurité SARSTEDT	37 - 40
HR	Uputa za upotrebu – SARSTEDT sigurnosna lanceta	41 - 44
HU	Használati utasítás – SARSTEDT biztonsági lándzsa	45 - 48
IT	Istruzioni d'uso – Lancetta Safety SARSTEDT	49 - 52
KO	사용 설명서 – SARSTEDT 안전 랜셋	53 - 56
LT	Naudojimo instrukcijos – SARSTEDT saugūs lancetai	57 - 60
LV	Lietošanas norādes – SARSTEDT drošības lancete	61 - 64
NL	Gebruiksaanwijzing – SARSTEDT Safety-lancet	65 - 68
NO	Bruksanvisning – SARSTEDT Safety-lansett	69 - 72
PL	Instrukcja obsługi – Bezpieczny nakłuwacz SARSTEDT	73 - 76
PT	Instruções de utilização – Lanceta de segurança SARSTEDT	77 - 80
RO	Instrucțiuni de utilizare – Lansetă de siguranță SARSTEDT	81 - 84
RU	Инструкция по применению – Безопасный ланцет SARSTEDT	85 - 88
SK	Návod na Použitie – SARSTEDT Bezpečnostná lanceta	89 - 92
SL	Navodila za uporabo – Varna lanceta SARSTEDT	93 - 96
SV	Bruksanvisning – SARSTEDT Safety Lancet	97 - 100
TH	คำแนะนำในการใช้งาน – ไขมีดนิรภัยของ SARSTEDT	101 - 104
TR	Kullanım için talimatlar – SARSTEDT Güvenlikli lanset	105 - 108
ZH	使用说明 – SARSTEDT安全采血针	109 - 112

Intended use

Safety-Lancets with safety feature are used for single use capillary blood sampling.


Specification

The products are intended for human use in a professional environment by qualified medical personnel.

Product description

The safety lancet is a sterile product for single use. The safety lancet consists of a plastic casing, a disposable needle/blade of stainless steel, a release button in various colors, steel springs and a safety plunger. The lancets contain a safety feature. After puncture the needle retracts automatically into the housing to prevent sharps injuries.

Safety lancet product line

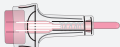Image	Version	Needle size	Puncture depth
	Mini	28 G	1.6 mm
	Normal	21 G	1.8 mm
	Extra	18 G	1.8 mm
	Super	Blade 1.5 mm	1.6 mm
	Neonatal	Blade 1.5 mm	1.2 mm

The different versions of the safety lancet are suitable for drawing different amounts of blood as required.

The selection of the correct version depends on various factors such as skin type, skin blood flow, puncture site and the correct placement of the safety lancet on the skin. If in doubt, testing several different safety lancets is recommended before routine use.

Safety instructions and warnings

1. For use only on a single patient. Discard the entire device after use.
2. Warning: Not intended for more than one use. Do not use on more than one patient. Improper use of blood lancets can increase the risk of inadvertent transmission of blood borne pathogens, particularly in settings where multiple patients are tested.
3. General precautions. Conduct hand hygiene before and after the process. Wear gloves and other general personal protection equipment to protect yourself from blood and possible exposure to pathogen transmissible by blood.
4. Treat all biological specimens and sharp blood collection utensils (needles, adapters) according to the guidelines and procedures in your institution. In case of direct contact to biological specimens or a needle prick injury, consult a doctor, as HIV, HCV, HBV or other infectious diseases can be transmitted. The safety guideline and procedures of your institution must be followed.
5. Dispose of the entire safety lancet in suitable sharp containers for sharp object after use.
6. Before using, check the safety plunger for damage and do not use if it is damaged.
7. Reusing the product can lead to injections, injuries or death.
8. Do not use the product after the expiry date. The expiry date of the product is the last day of the month and year indicated.

Storage

Store the product at room temperature.

Handling

READ THIS DOCUMENT COMPLETELY BEFORE STARTING THE PUNCTURE.

Materials needed for collecting a specimen:

1. All necessary sample containers, e.g. micro sample tubes, labelled by size and additive as per request.
2. Gloves, lab coat, eye protection or other suitable protective gear to protect against pathogens transmitted by blood or potentially infectious material.
3. Labels for sample identification.
4. Safety lancet.
5. Disinfectant for disinfecting the puncture site (follow the facility's guideline for preparing the puncture site for taking blood samples). Do not use any cleaning material containing alcohol if the specimens are to be used for a blood alcohol test.
6. Dry, germ-free swabs.
7. Plaster or dressing.
8. Sharps disposal container for the safe disposal of used material.

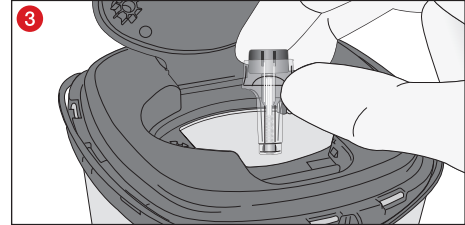
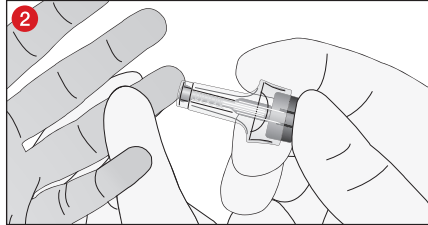
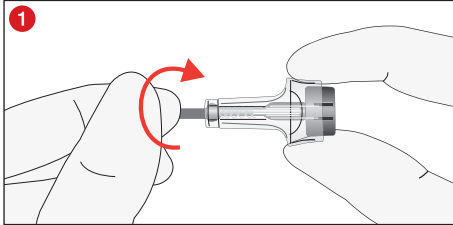
NOTE: Follow the regulations of your institution for the order of collection.

Puncture with the safety lancet

General instructions:

WASH HANDS BEFORE AND AFTER USE OF THE SAFETY-LANCET. WEAR GLOVES TO MINIMISE THE RISK OF EXPOSURE DURING CAPILLARY BLOOD COLLECTION.

1. If needed, select a suitable sample tube or prepare a POC test for the required specimen.
2. Select and disinfect a site for the puncture with a safety lancet. Please follow the guidelines of the institution for preparing the puncture site.

- 1 Twist the protective cap off.
- 2 Hold the safety lancet against the selected and disinfected puncture site. Press the release button. After use the lancet will automatically retract into the housing.
- 3 Dispose of the safety lancet in a suitable container for sharp objects.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Dispose of the safety lancet in a sharps container.
3. Disposable gloves prevent the risk of infection.
4. Disposal of potentially contaminated consumables in accordance with institutional guidelines.

US standards / ISO standards in the latest version

Standards for labelling:

DIN EN ISO 15223-1: Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements

Standards for needles/cannulas:

DIN EN 10088-1, Table 2 - Stainless steels - Part 1: List of stainless steels

Standards for sterilisation:

DIN EN ISO 11137-1: Sterilization of health care products - Radiation - Part 1: Requirements for development, validation and routine control of a sterilization process for medical devices.

DIN EN ISO 11137-2: Sterilization of health care products - Radiation - Part 2: Establishing the sterilization dose

System-related standards:

DIN EN ISO 13485: Medical devices - Quality management systems – Requirements for regulatory purposes

DIN EN ISO 14971: Medical devices - Application of risk management to medical devices

CLSI* GP42 "Collection of Capillary Blood Specimens", Approved Standard.

*CLSI (Clinical and Laboratory Standards Institute)

MDSAP: *Medical Device Single Audit Program* in accordance with ISO 13485

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