

*To make your
workflow excellent.*



ESR System Solutions

For venous and capillary blood collection:

Sediplus® S 2000 NX, S-Sedivette®, Microvette® CB 200 ESR and accessories



Pre-analytics

S-Sedivette®

Closed system from blood collection to erythrocyte sedimentation

The S-Sedivette® allows proven closed venous blood collection with choice of aspiration or vacuum technique. 2.8 ml blood are added to the S-Sedivette® pre-dosed with 0.7 ml citrate (mixing ratio 1:4).




The ESR measurement is carried out directly in the S-Sedivette® as a closed system.

- No risk of infection by transferring blood
- Sedimentation measurement directly in the sampling tube
- No sedimentation pipette needed
- Easy handling
- Economical and time-saving

Sedimentation rack for the S-Sedivette®

The SARSTEDT ESR rack with graduated scale is optimised for manual ESR measurement.

- 10 samples can be measured at the same time
- Measurements independent of each other in time
- Sedimentation result can be read after 1 hour
- Parallax-free reading
- Measurements based on the mm values using the Westergren method

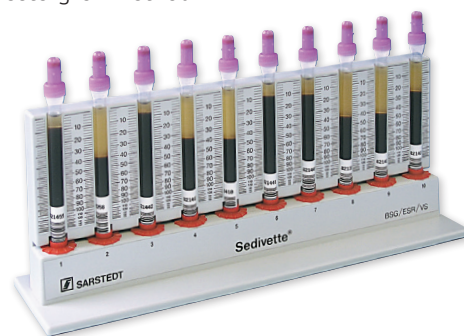

Order no.	Name
06.1690.001	S-Sedivette® 50 pieces in cardboard packaging, 500 pieces in shipping box
90.1090	S-Sedivette rack with graduated rear wall for 10 S-Sedivettes

Microvette®

Microvette® CB 200 ESR and ESR rack


Measurement of erythrocyte sedimentation from capillary blood

The Microvette® CB 200 ESR pre-dosed with citrate contains a stopper with assembled and prepared End-to-End collection capillary and a sedimentation capillary.

- Verified good comparison values with the Westergren method
- Designed specifically for capillary blood collection
- Minimal patient discomfort due to collection volume of only 200 µl

ESR rack

- The ESR rack with graduated rear wall and 10 measuring stations is designed specifically for the Microvette® CB 200 ESR.

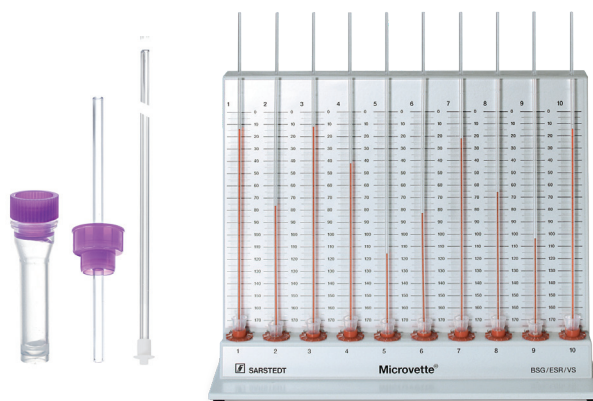

Order no.	Name
18.1325	Microvette® CB 200 ESR
90.1091	ESR rack for Microvette® CB 200

Sediplus®

Sediplus® S 2000 NX · Automatic 40-channel blood sedimentation measuring device

Based on SARSTEDT's many years of experience, the Sediplus® S 2000 NX is the new generation of our proven automatic 40-channel blood sedimentation measuring device.

The modern touch display and intuitive menu navigation ensure easy, user-friendly handling.

- Combines safe blood sampling with reliable ESR determination
- 40 samples can be measured at the same time
- Measurements independent of each other in time
- Simple operation and automatic reading of the erythrocyte sedimentation
- Measurements based on the mm values using the Westergren method

The 1 h and 2 h, optionally the ½ h and 1 h value are measured and a complete log is sent to the central EDP after completing the measurement. There is no need to monitor sedimentation.


Technical data:

Measuring principle:	IR transmission
Measuring range:	0–65 mm absolute, corresponding to 0–116 mm Westergren
Measurement accuracy:	+/- 1 mm measuring distance
Display:	mm Westergren
Measuring stations:	40 positions
Data storage:	Date, time, position, sample ID, measured values, error messages if applicable
Data output:	Display and HL7 standard to EDP or printer
Power supply:	110–230 V, 50–60 Hz, 16 W
Dimensions (WxDxH):	270x355x305 mm
Weight:	6.7 kg
Measuring times:	Measured value 1: ½ h and measured value 2: 1 h Measured value 1: 1 h and measured value 2: 2 h
Display:	Touch display
Functional test:	Test Sedivettes included

Order no.	Name
90.1092	Sediplus® S 2000 NX
90.1092.720	Thermal printer
90.1092.730	Barcode scanner for Sediplus® S 2000 NX

Test Sedivettes

For functional testing of the Sediplus® S 2000 NX. Following a fixed procedure, the functional elements relevant to the measurement (mechanics and optics) and thus the proper functioning of the device can be checked channel by channel. The test Sedivettes are included with the device.

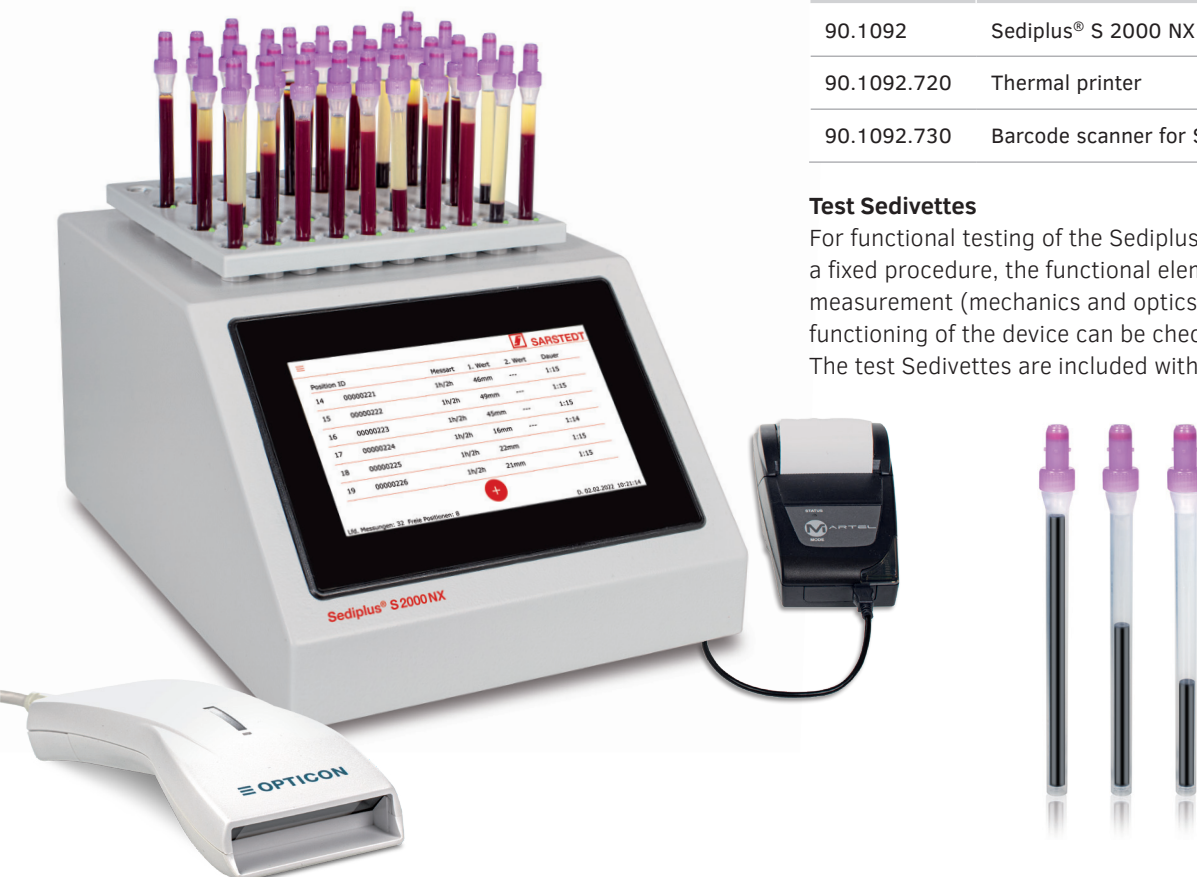

If you have any questions:
We'd be happy to help!

Besuchen Sie auch unsere Internetseite:
www.sarstedt.com

SARSTEDT AG & Co. KG

Sarstedtstraße 1
D-51588 Nümbrecht

Phone: +49 2293 305 0

export@sarstedt.com
www.sarstedt.com



Further information,
videos and sample order:
sarstedt.com