

Single Patient Use Tourniquet

Designed for efficient infection prevention



- Patient-specific for multiple use
- Very soft, latex-free material for enhanced patient comfort
- Intuitive, easy handling
- For paediatric and adult use

Handling of Single Patient Use Tourniquet

One-hand technique for easy handling

TourniSTRETCH® was developed to provide efficient infection prevention.

Made from a customised material, the TourniSTRETCH® is strong and flexible for creating optimal tension yet feels soft and comfortable to patients. It features an intuitive pull through system that allows for re-adjustment and quick release for easy application and enables single-handed operation during blood collection.



User Guide



Wrap TourniSTRETCH® around the limb with peg side up



Feed end through the buckle to create sufficient tension



Pull end back over the peg to secure in position



For further information please visit

www.sarstedt.com/tournistretch

Ordering information

Art. No.	Description	Packaging
95.1008	TourniSTRETCH® Single Patient Use Tourniquet	25 pcs. / bag 500 pcs. / box

SARSTEDT Ltd.
Optimus Way
Optimus Point
Leicester LE3 8JR
Tel: +44 116 235 9023
Fax: +44 116 236 6099
info.gb@sarstedt.com
www.sarstedt.com