


S-Monovette® Citrat

Wichtige Information zur Handhabung

Zur Vermeidung von Fehlmessungen und Nichtbearbeitung der Proben aufgrund von Unterfüllungen ist ein **exaktes Füllvolumen** und damit **korrektes Mischungsverhältnis (1:10)** zwingend erforderlich.


Diese Fehler lassen sich vermeiden, wenn die S-Monovette® Citrat bis zum Anschlag aufgezogen wird. Danach noch ca. 5 sec. warten, bis der Blutfluss sichtbar stoppt.

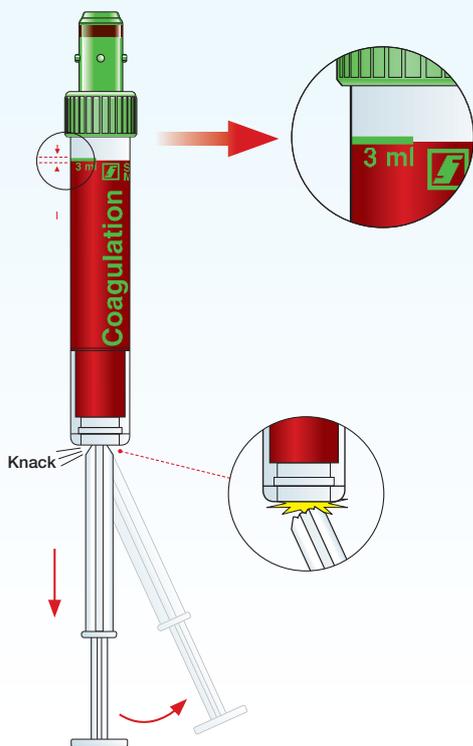

Barcode-Etikettierung

richtig

falsch

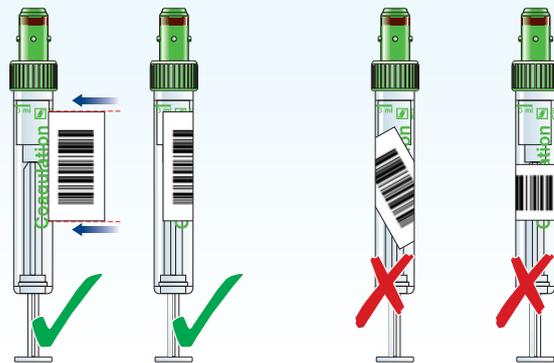


Konnektierung

S-Monovette® zur Konnektierung mit Safety-Kanüle oder Safety-Multifly® **immer** **aufrecht** halten.

richtig

falsch

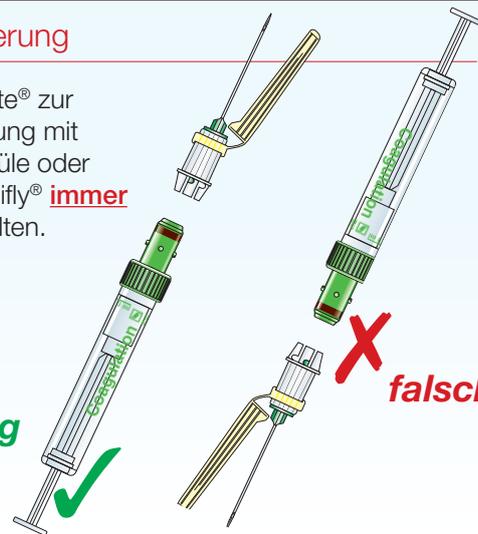

Blutentnahme der S-Monovette® Citrat mit Safety-Multifly®-Kanülen


Bei Verwendung eines Safety-Multifly®-Sets sollte die S-Monovette® Citrat **nie als 1. Röhrchen** verwendet werden, da durch die Luft im Schlauch (20 cm Länge = 300 µl) eine Unterfüllung die Folge wäre.



Sollte ausschließlich eine S-Monovette® Citrat eingesetzt werden, immer erst eine andere S-Monovette® (z.B. Serum) zum „Befüllen des Schlauches“ davor setzen!

Wichtig: Gut mischen!

Die Blutprobe direkt nach der Entnahme **mindestens 3x** über Kopf schwenken!

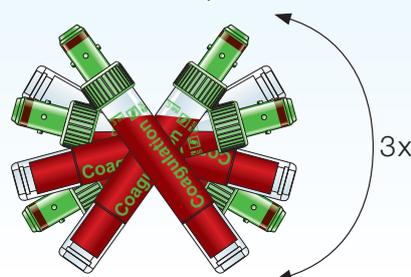


SARSTEDT AG & Co. KG
Postfach 12 20 · D-51582 Nümbrecht
Telefon: +49 2293 305 0
Telefax: +49 2293 305 3450
Kundenservice Deutschland
Telefon 0800 0 83 305 0
info@sarstedt.com
www.sarstedt.com

S-Monovette® Citrate

Important Information for Use

To avoid incorrect measurements and non-processing of samples due to underfilling, an **exact filling volume** and therewith **correct mixing ratio (1:10)** is absolutely necessary.


These errors can be avoided if the S-Monovette® Citrate is pulled up to the stop. Then wait approx. 5 seconds until the blood flow visibly stops.

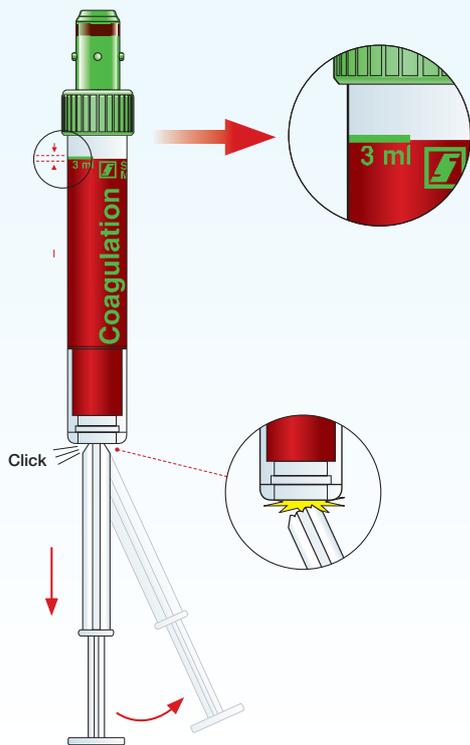

Barcode Labelling

correct

wrong

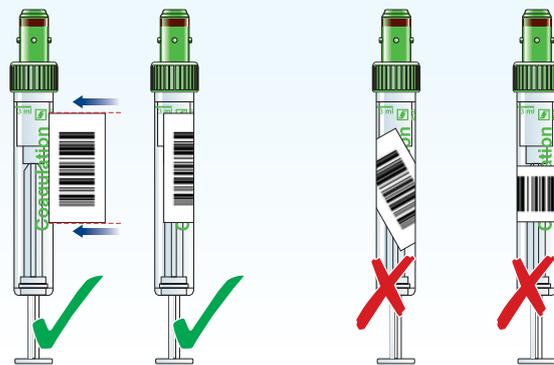


Connection

Always keep the S-Monovette® **upright** for connection with a Safety-Needle or Safety-Multify®.

correct

wrong

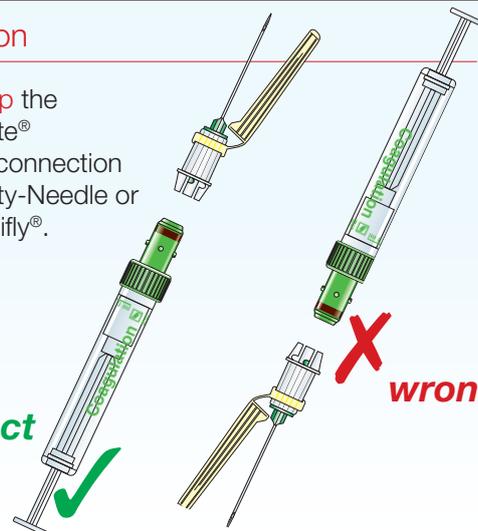

Blood collection of the S-Monovette® Citrate with Safety-Multify®-Needles

When using a Safety-Multify® set, the S-Monovette® Citrate should **never be used as the first tube**, as the air in the hose (20 cm length = 300 µl) would result in underfilling.

If only one S-Monovette® Citrate is used, always use another S-Monovette® (e.g. Serum) in front of it to fill the Safety-Multify® tubing!



Important: Mix well!

Invert the blood sample at least **3 times** immediately after collection!

