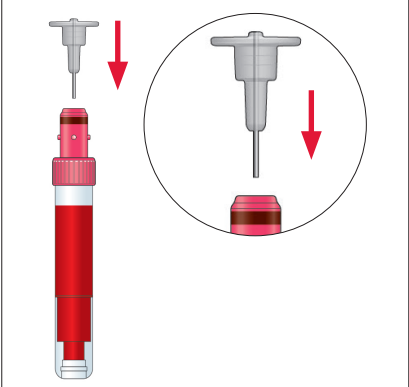
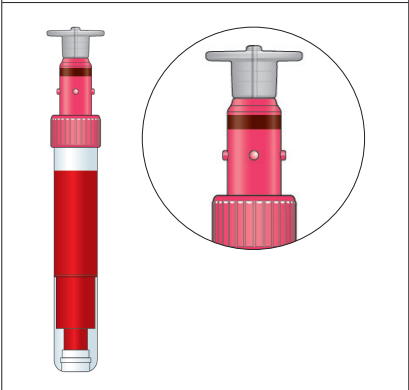
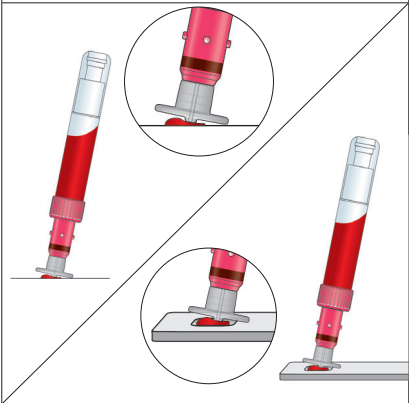
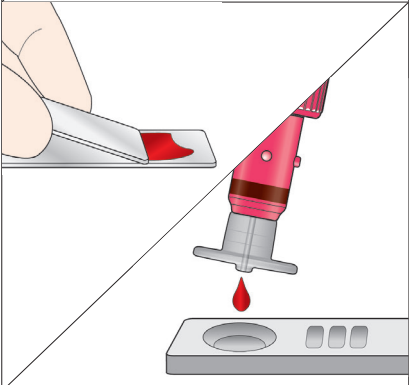


# Handhabung HaemoDiff® 14.1217





	<p>1. Setzen Sie das Haemo-Diff® mit leichtem Druck senkrecht auf eine mit Blut befüllte S-Monovette® auf.</p>
	<p>2. Die Spitze des Haemo-Diff® durchstößt die Membran der S-Monovette®. In diesem Stadium kann kein Blut austreten.</p>
	<p>3. Tippen Sie das Haemo-Diff® unter leichtem Druck auf einen Objektträger, so dass ein Blutstropfen auf die Glasoberfläche abgegeben wird.</p> <p><b>Oder</b></p> <p>Für die Durchführung eines POCT Test kann das Blut unter leichtem Druck als Tropfen in die Probenvertiefung der POCT Testkarte gegeben werden.</p>
	<p>4. Zur Durchführung des Blutabstrichs selbst verfahren Sie wie gewohnt oder auf die übliche Weise.</p> <p><b>Oder</b></p> <p>Für die Durchführung eines POCT Tests befolgen Sie die Anweisung des jeweiligen Testkit Herstellers.</p>

Technische Änderungen vorbehalten

Diese Publikation kann Informationen zu Produkten enthalten, die evtl. nicht in jedem Land verfügbar sind

SARSTEDT AG & Co. KG  
Sarstedtstraße 1 · D-51582 Nümbrecht  
Telefon: +49 2293 305 0  
Telefax: +49 2293 305 3450  
Kundenservice Deutschland  
Telefon 0800 0 83 305 0  
info@sarstedt.com  
www.sarstedt.com

# Handling HaemoDiff® 14.1217

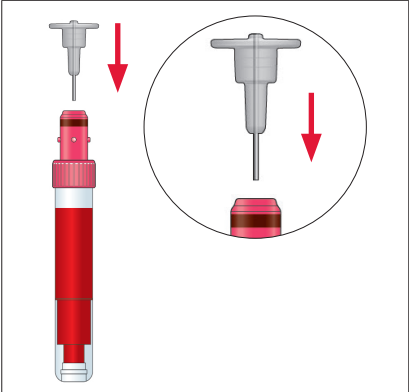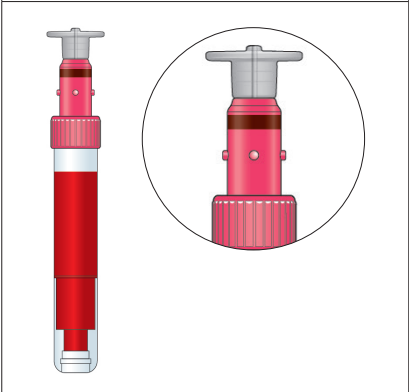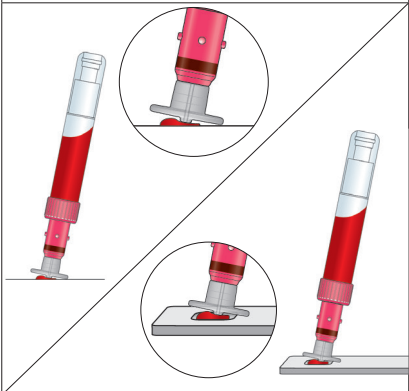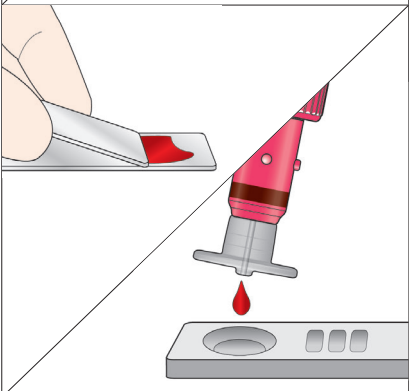	<p>1. Attach the Haemo-Diff® vertically onto the S-Monovette® applying gentle pressure.</p>
	<p>2. The spike on the Haemo-Diff® pierces the membrane of the S-Monovette®. No blood can leak at this stage.</p>
	<p>3. Tap the Haemo-Diff® onto a slide under light pressure so that a drop of blood is released onto the glass surface.</p> <p><b>or</b></p> <p>To perform a POCT test, the blood should be added as drops to the specimen well of the POCT test card under slight pressure.</p>
	<p>4. To perform the blood smear itself, proceed as usual or in the usual way.</p> <p><b>or</b></p> <p>To perform a POCT test, follow the instructions of the respective test kit manufacturer.</p>

Technical modifications reserved

This document may contain information on products that may not be available in particular countries

BFO\_15\_957\_0000\_142

SARSTEDT AG & Co. KG  
Sarstedtstraße 1  
D-51582 Nümbrecht  
Phone: +49 2293 305 0  
Fax: +49 2293 305 3992  
export@sarstedt.com  
www.sarstedt.com