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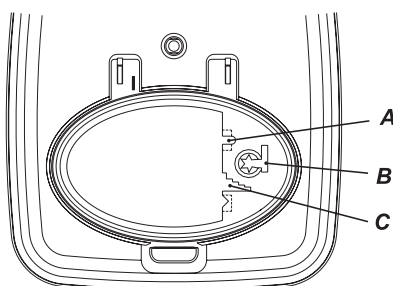
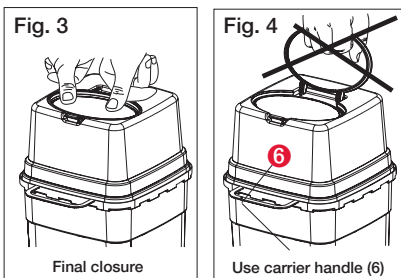
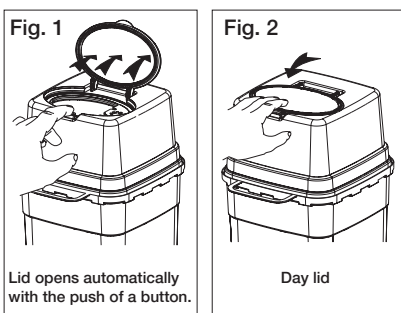
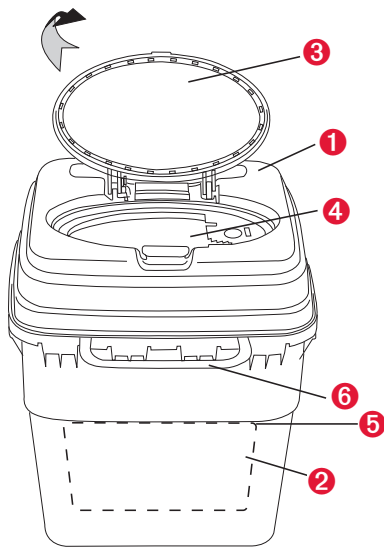
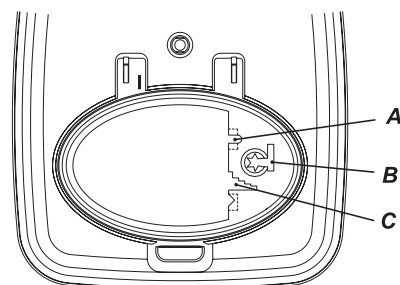
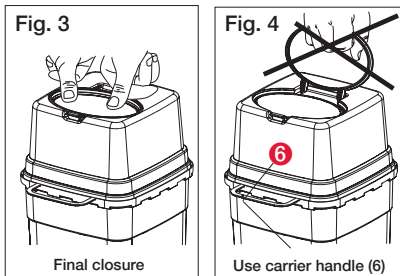
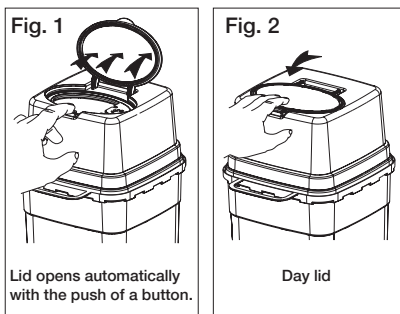
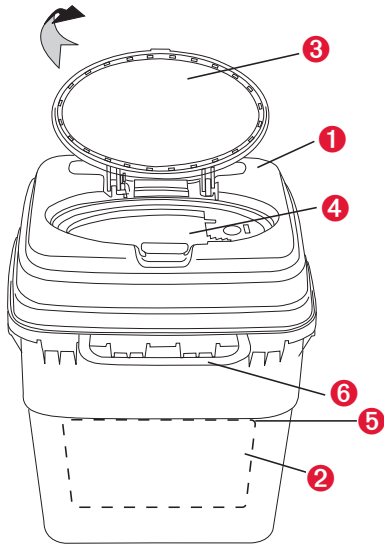
# Instructions for Use: Multi-Safe euroMatic®

## 1. Intended purpose

The disposal box is exclusively intended for the disposal of pointed and sharp items like, for example, needles and scalpel blades for medical use.

## 2. Safety instructions

- Use only boxes adequately sized to accommodate the items to be disposed.
- Keep the box beyond the reach of children.
- Ensure that the lid (1) is assembled and locked into place before filling.
- Attach the box to recommended base adapter or wall holder, or place into the Safety-Tray to protect it against overturning.
- Ensure that items are placed into the box in a vertical direction only.
- Do not use the day lid (3) to force objects into the box.
- Ensure that scalpels are placed into the box with special care. Excessive force used to place sharps or subsequently fill other objects into the box involves the risk of jamming or damage to the walls or base.
- Do not use force to press objects into the box.
- Do not fill with liquids.
- Do not fill the box above the maximum filling line (5) (health hazard!).
- Do not touch the inside of the box with your hands or otherwise (health hazard!).
- Do not throw.
- Do not shake.
- Do not drop.
- Ensure that no objects are jutting out prior to closing the box.
- Do not use force to close the box and close by hand only.
- Do not press the box against your body (e.g. chest) for final closure.
- Do not press the box together from the outside when full.
- Use the carrier handle for transport.
- Prior to disposal, ensure that lids 1 and 3 are securely closed.

## 3. Assembly (lid assembly)

Prior to filling, place the box on a solid surface with the carrier handle (6) facing in your direction. Place the lid (1) squarely onto the base (2) and push down the opposite sides of the lid (1) with both hands. Placed on a table, be aware that you may not be able to exert sufficient force due to the height of the box.

## 4. Daily use

The box opens automatically by pressing the push button (Fig. 1). For daily use, press the day lid (3) lightly into the filling opening (4) to enable re-opening (Fig. 2).

## 5. Final closure of day lid (3)

After filling, proceed as follows: Place the box on a solid surface and press the day lid (3) firmly into the filling opening (4) until the day lid audibly locks into place along the entire circumference of the box (Fig. 3).

Press the pushbutton to release a yellow ring indicating that the box must be transferred to the disposal site.

## 6. Fill line - maximum filling level (5)

The upper section of the box features a fill line (5). Ensure that the box is not filled above this line (5).

For safety reasons, do not fill the box above the maximum filling level (5).

### 7.1 Holding fixtures

A base adapter and wall holder are available to protect the box against unintentional overturning (Refer to separate User Guide: "Holding Fixtures").

### 7.2 Carrier handle (6)

The upper section of the box features a carrier handle (6). After final closure of the disposal box (Fig. 3), be aware to hold the box by this handle (6) for convenient transport to the disposal site (max. 6 kg, approx.).

## 8. Removal aids

### 8.1 U-shaped recess (A)

for sharps removal from Luer lock systems.

### 8.2 Slotted recess (B)

For the removal of scalpels.

### 8.3 Stepped recess (C)

for the removal of needles.

## 9. Disposal and identification

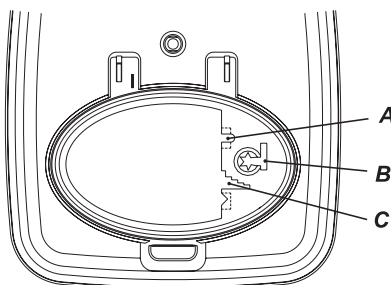
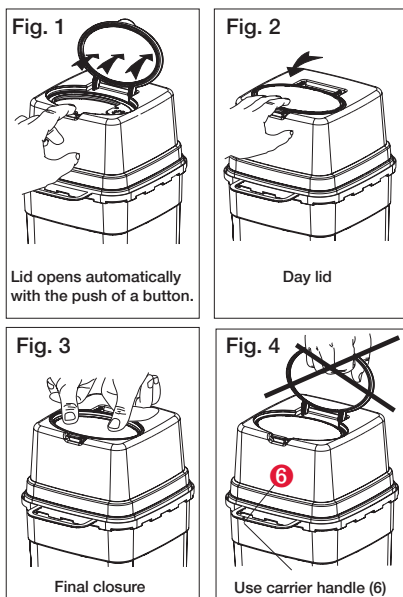
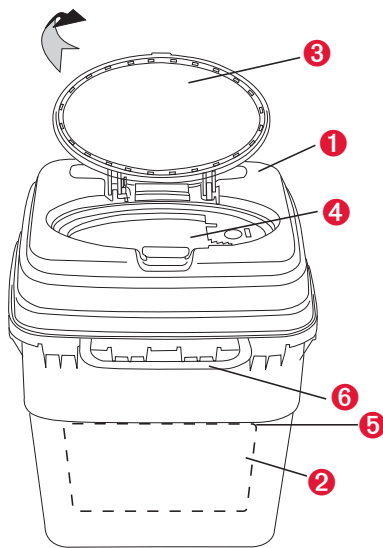
Disposal boxes are made from ecologically friendly polypropylene. Incineration does not cause emission of any harmful substances. Observe statutory provisions pertaining to disposal!

## 10. DIN EN ISO 23907

If you use the disposal box in agreement with this directive, you will have to insert one of these absorbent liners:

Packaging	Order number
1.000	78.669
	for 6 l and 7 l version:
100	95.1712

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