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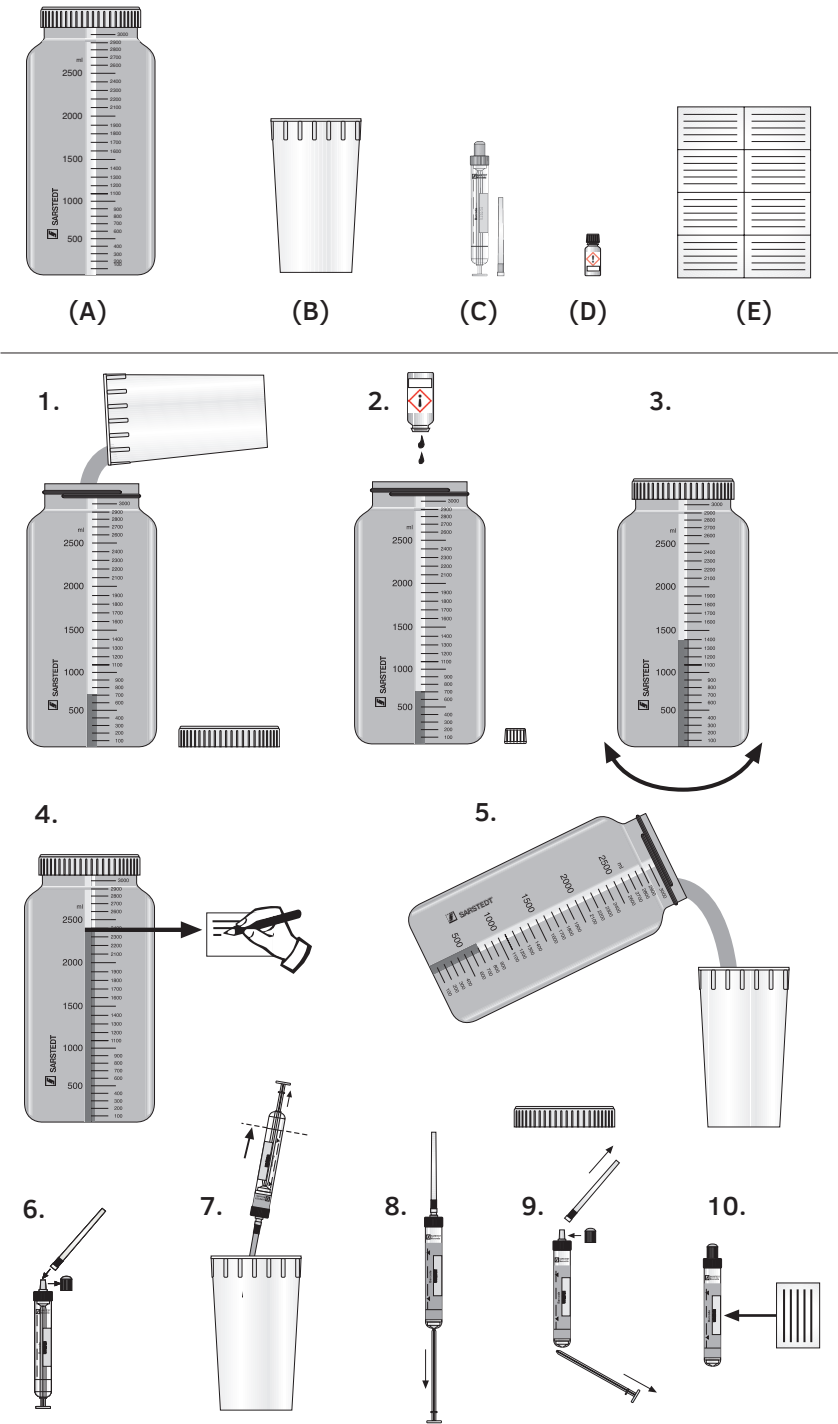
This is what you received:
Questa lei la ricevuto:

EN Instructions for Use

UriSet 24 with Urine Monovette®

After getting up in the morning empty urinary bladder into the toilet. Write down the date and time on the enclosed label (E).

- Please use the receptacle (B) for the following mictions and fill this urine into the urine container (A).
- Then drip the complete contents (acid) from the small glass bottle (D) into the urine container (A).

!

H 315
H 319
H 335

The small glass bottle contains 9 ml hydrochloric acid 20 %

Irritation to eyes, respiratory organs and skin. Keep tightly closed and inaccessible for children. In case of contact with eyes, flush with plenty of water immediately and consult a medical doctor. In case of accident or, if indisposed, seek medical advice immediately (if possible show this label).

Attention

- Close the urine container (A) well and shake it. Collect any further urine, also in case of bowel movement, and fill it into the urine container (A). Close the urine container (A) well after every use and shake it. The next day, go to the toilet at the time noted down the day before and collect this urine. Collection is completed now.
- Read the collected quantity from the scale on the urine container (A). Write down this figure on the label (E). Fill in the label (E) completely.
- Fill some urine from the urine container (A) into the receptacle (B).
- Remove the stopper from the Urine Monovette® (C) and keep for later use. Attach the suction tip to the Urine Monovette® (C).
- Insert the suction tip into the receptacle (B) and fill the Urine Monovette® (C) up to the base line.
- To empty the suction tip, hold the Urine Monovette® (C) in a vertical position, away from the receptacle (B), and withdraw the piston rod in a downward direction until it locks into its base.
- Remove the suction tip and break off the piston rod. Replace the stopper. Empty the remaining urine from the receptacle (B) and the urine container (A) into the toilet.
- Attach the completed label (E) to the Urine Monovette® (C). Hand the filled tube (C) over to your physician. All components of the urine collection set can be disposed of as domestic waste.

(A): Urine Container (B): Receptacle (C): Urine Monovette®
(D): Glass bottle contains 9 ml hydrochloric acid 20 % (E): Label for Urine Monovette®

Store at room temperature Technical modifications reserved

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