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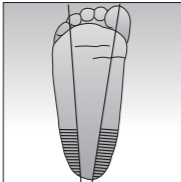
Intended use

The heel incision safety lancet is mainly used to collect the blood from the heels of newborn babies for blood test. It is used for blood sampling from heels of infants during blood tests.

Materials Required But Not Provided

- Antiseptic swabs, 70% Isopropanol
- Sterile gauze pads
- Blood collection container, capillary tube or similar device

Summary of Procedure
This summary of procedure is only valid in combination with the user guide on the inner box. The so-called safe area to inflict a neonatal or infant heel is located on the plantar surface medial to a line drawn posteriorly from the middle of the big toe to the heel or lateral to a line drawn posteriorly from between the fourth and fifth toe to the heel. Illustrated below.

With the baby in a supine position, the heel incision site is cleaned with an antiseptic swab and air dried. The Safety-Heel® Incision Safety Lancet is properly positioned on the heel and triggered. The first drop of blood is wiped away using a dry sterile gauze pad. The blood specimen is then collected in the desired capillary tube or container. Following blood collection, gently press a sterile pad to the incision site until bleeding has ceased. The baby's heel must then be monitored for late bleeding and inflammation.

Conditions Affecting the Procedure

- Heel oedema.
- Re-incision of prior wound site.
- Swollen /Inflamed heel.
- Excessive pressure and skin indentation from placing the device on the heel, resulting in deep and hazardous wound depths.
- Poor vascularisation may cause inadequate blood flow.

Warning

- Do not repeat blood sampling at the same incision site.
- Do not use in infants with edema, infection or inflammation at the incision site.
- Do not use in infants with skin diseases or skin lesions at the incision site.

Cautions
1. Do not use if the individual packaging has been previously damaged
2. Check the use-by date on the packaging, and do not use the lancet beyond the use-by date.
3. The safety lancet is for disposable use and do not reuse.
4. Discard the used lancet into a suitable sharps container.

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