

# S-Monovette®


## Recommendations on Centrifugation

	S-Monovette® Serum	10 min.	2,000 x g	20°C
	S-Monovette® Serum-Gel*	10 min.	2,500 x g	20°C
	S-Monovette® Li-Heparin	10 min.	2,000 x g	20°C
	S-Monovette® Li-Heparin-Gel* or	10 min. 15 min.	3,000 x g 2,500 x g	20°C 20°C
	S-Monovette® EDTA-Gel*	10 min.	2,500 x g	20°C
	S-Monovette® Citrate	10 min.	1,800 x g	20°C

\*For S-Monovettes with gel preparation we recommend to use swing-out rotors only.

Refer to the nomogram on the back or the centrifugation calculator at [www.sarstedt.com/User Information/Centrifugation](http://www.sarstedt.com/User Information/Centrifugation) for conversion of g-force into rotational speed/min.

Nomogram for conversion of g-force into rotational speed/min

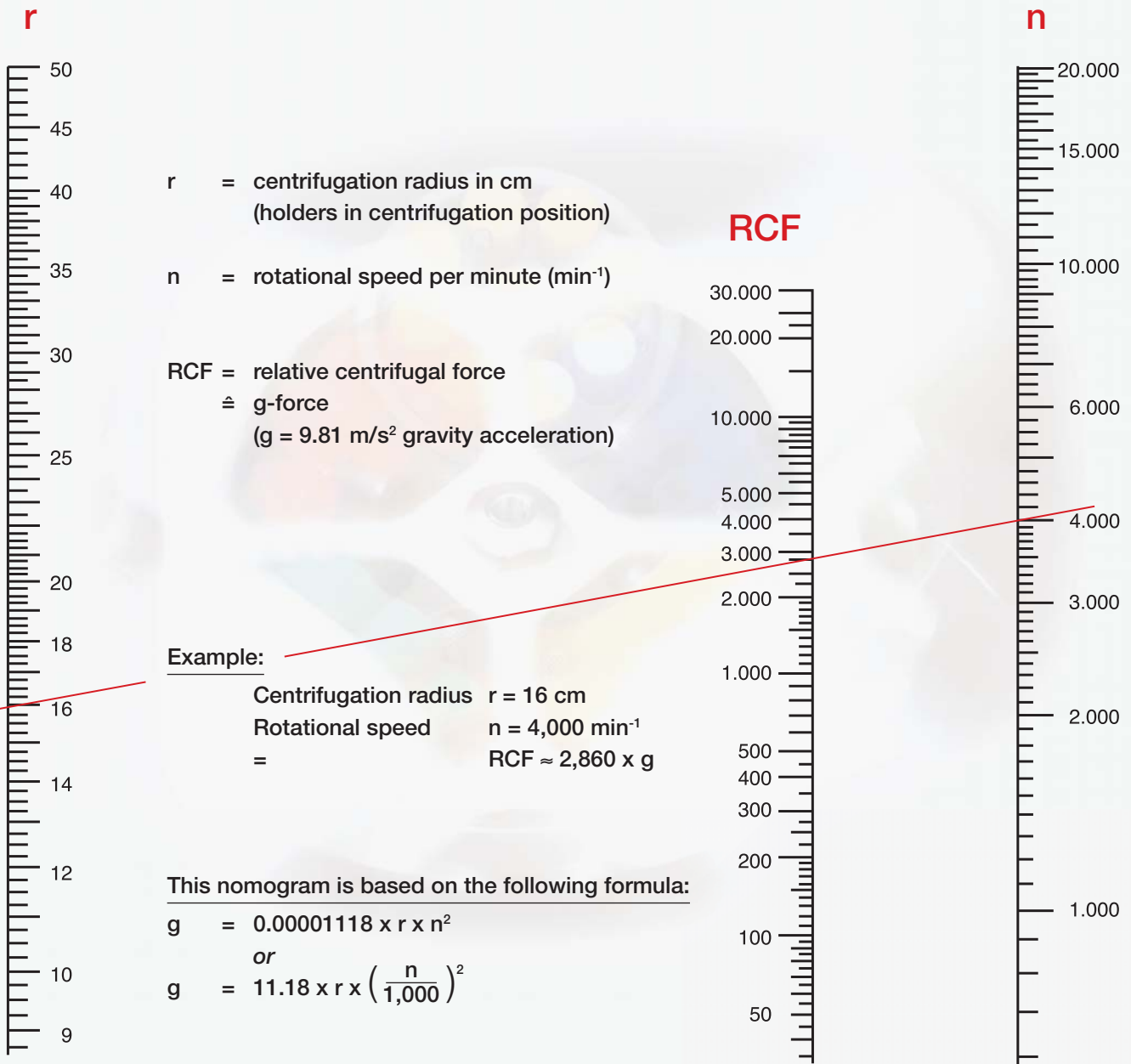

Technical modifications reserved

This document may contain information on products that may not be available in particular countries

427/a-500-0610-0413

SARSTEDT AG & Co.  
P.O. Box 12 20  
D-51582 Nümbrecht  
Phone +49 2293 3050  
Fax +49 2293 305- 122  
info@sarstedt.com  
www.sarstedt.com